

# BACtrack® Select S70 Breathalyzer Test

Distributed by  
**CLIAwaived™ Inc.**



The BACtrack Select Breathalyzer S-70 Test is an innovative device designed to quickly and easily test blood alcohol levels. Equipped with several features, the BACtrack Select S-70 Test provides users with accurate and reliable results; to use, simply insert the mouthpiece and begin testing immediately. The proprietary FlowCheck technology ensures that a continuous deep lung air sample is taken at every test. Included are six replaceable, hygienic mouthpieces, each individually wrapped for safe storage prior to use; the mouthpieces allows the device to become a safe and sanitary for testing multiple users.

#### BENEFITS:

- Simple to use, compact and portable
- Easy to view LCD display
- Wide test range with professional accuracy
- EasyView provides test alerts to user
- FlowCheck ensures a continuous deep lung air sample
- Replaceable, hygienic individually wrapped mouthpiece
- Safe and sanitary for testing multiple users



<b>Ordering Information:</b>	
Product Description	Catalog #
BACtrack® Select S70 Breathalyzer Test (Breathalyzer, 6-mouthpieces, case, 2-“AA” batteries)	BAC-S70
BACtrack® Select Series Mouthpieces (10 pack)	BAC-MPS-10
BACtrack® Select Series Mouthpieces (50 pack)	BAC-MPS-50
<b>Reimbursement Information:</b>	
CPT Code #	National Limit Amount
N/A	N/A

All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

# BACtrack® Select S70 Breathalyzer Test Specifications

## SPECIFICATIONS

Dimensions	2.3 x 4.8 x 0.8 inches (5.8 x 12.2 x 2.2 cm)
Weight	4.2 oz (119 g) with mouthpiece and batteries
Sensor Technology	Advanced semiconductor
Detection Range	0.000 – 0.400 %BAC
Test Count	Displays total number of tests performed
Power Supply	Two AA alkaline batteries, included
Battery Life	Approximately 300 tests
Warm Up Time	10-30 seconds, depending on last use
Response Time	3 seconds
Sensor Accuracy	+/- 0.01 %BAC @ 0.02 %BAC
Operating Temperature	50-104 °F (10-40 °C)

This device has been tested using NHTSA/DOT procedures. In this testing, the accuracy of this device was established at Blood Alcohol Concentrations (BAC) of 0.008 and 0.032, but not at greater concentrations. Additional laboratory testing has been performed at concentrations ranging from 0.00 – 0.40% BAC.

**Warranty:** Limited 1 year warranty on labor and parts

**Return policy:** Due to hygiene purposes, this item is non-returnable.

## PRECAUTIONS

1. Wait at least 20 minutes after drinking, eating or smoking before testing. Not observing this waiting period can cause inaccurate readings and damage the sensor.
2. Do not blow smoke, food, or liquids into the BACTRACK as this will damage the sensor.
3. Do not test in areas with strong winds, smoke, or in an area where a large amount of alcohol is being consumed.
4. Replace the two AA batteries when the battery indicator icon reaches one bar.
5. Send your tester in for periodic calibration service as required. (See calibration).
6. The BACTRACK Breathalyzer is designed to be used in a temperature range of 50-104 °F (10-40 °C). Operation of the unit in temperature ranges above or below this range may affect the accuracy of results.
7. Avoid testing in the presence of any substances that contain methyl alcohol, isopropyl alcohol or acetone. These substances may interfere with the results of the test.
8. You cannot use the results of this product in court.
9. Do not use the BACTRACK as a tool to determine whether you should operate a motor vehicle or equipment, or perform any other dangerous act.
10. Do not drink and drive. Always have a designated driver when alcohol is being consumed.