


CLIAwaived™ Inc. Rapid Strep A Test

CLIAwaived™ Inc. Rapid Strep A Test

The CLIAwaived™ Inc. Rapid Strep A is a lateral-flow, membrane-based immunoassay for the qualitative detection of Group A Streptococcal antigen from throat swab specimens. Group A Strep is one of the most serious causes of upper respiratory infections, and the major cause of pharyngitis which, if left untreated, can lead to serious complications such as rheumatic fever, acute sinusitis, scarlet fever and glomerulonephritis. This test is for waived labs with a current Certificate of Waiver.

BENEFITS:


- Detects Strep A
- No cross reactivity with saliva
- 30 tests per kit (dipsticks)
- Results in 5 minutes
- Easy to read
- Fast, easy and reliable
- 18 month shelf life
- Room temperature storage
- FDA Cleared and CLIA waived

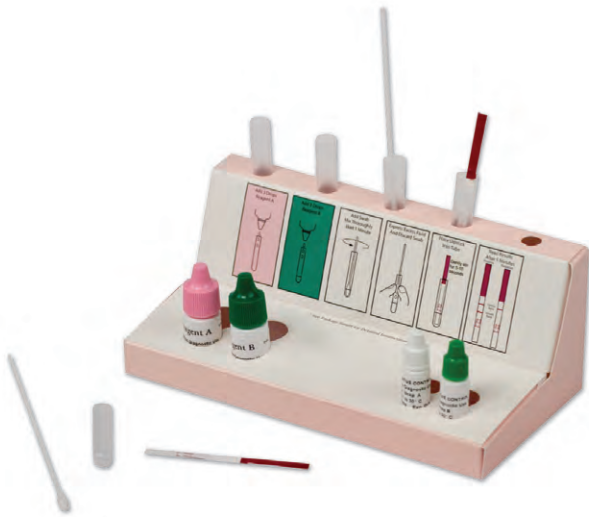

Ordering Information:

Product Description	Catalog #
CLIAwaived™, Inc. Rapid Strep A Dipstick Test Kit (30 tests)	CLIA-ST-501
Reimbursement Information:	
CPT Code #	National Limit Amount
87880QW (Strep A)	\$16.76

Test Procedure

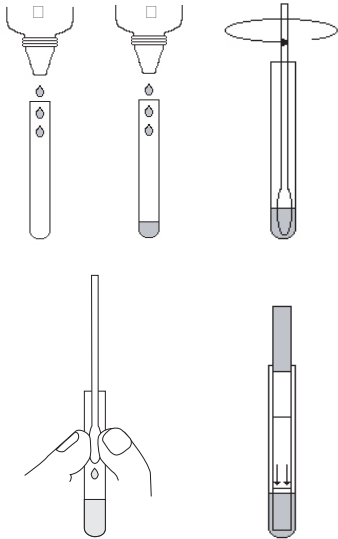

1. Add 3 drops of Reagent A, and 3 drops of Reagent B to the Extraction Tube.
2. Place the specimen swab into the extraction tube and mix the solution thoroughly by swirling the swab vigorously. Allow the swab to soak for at least 1 minute but no longer than 15 minutes. Remove all liquid from the swab by rolling it against the side of the extraction tube and then squeezing the swab between the flexible sides.
3. Place dipstick into extraction tube. Gently stir for 10-15 seconds.

Test Results



Read results after 5 minutes and no later than 10 minutes. One line indicates negative and two lines indicate positive.

(See package insert for detailed instructions.)

