

CLIA - 10 URINE REAGENT STRIPS 100 STRIPS - 10 PARAMETERS

FOR IN VITRO DIAGNOSTIC USE
CLIAwaived, Inc.

TESTS AND READING TIME	1	2	3	4	5	6	
LEUKOCYTES 2 min.	NEGATIVE 0	TRACE 15	SMALL (+) 70	MODERATE (++) 125	LARGE (+++) 500 ca CELLS/ μ L		
NITRITE 60 sec.	NEGATIVE	POSITIVE (ANY DEGREE OF PINK COLOR)					
URO-BILINOGEN 60 sec.	0.2 NORMAL 3.2	1 16	2 32	4 64	8 128	EHRlich UNITS/dl URINE μ mol/L	
PROTEIN 60 sec.	NEGATIVE	TRACE	30 (+) 0.3	100 (++) 1.0	300 (+++) 3.0	>2000 (++++) >20 mg/dl g/L	
PH 60 sec.	5.0	6.0	6.5	7.0	7.5	8.0	8.5
BLOOD 60 sec.	NEGATIVE 0	NON-HEMOLYZED 10 TRACE	HEMOLYZED TRACE	SMALL (+) 25	MODERATE (++) 80	LARGE (++) 200 ca CELLS/ μ L	
SPECIFIC GRAVITY 45 sec.	1.000	1.005	1.010	1.015	1.020	1.025	1.030
KETONE 40 sec.	NEGATIVE	TRACE (5) 0.5	SMALL (15) 1.5	MODERATE (40) 4.0	LARGE (80) 8.0	LARGE (160) 16 mg/dl mmol/L	
BILIRUBIN 30 sec.	NEGATIVE	SMALL +	MODERATE ++	LARGE +++			
GLUCOSE 30 sec.	NEGATIVE	100 5	250 (+) 15	500 (++) 30	1000 (++) 60	> 2000 (++) 110 mg/dl mmol/L	

Distributed by: **CLIAwaived.com™**
San Diego, CA 92121
tel 858-481-5031
toll free 888-882-7739
www.cliawaived.com



IMPORTANT: READ PACKAGE INSERT BEFORE USE.
DO NOT touch test areas. Store at temperature between
15°-30° (59°-86°F). DO NOT remove desiccants.
Be sure to replace cap immediately and tightly. V/B

DATE OPENED _____ DO NOT USE AFTER 90 DAYS OF BREAKING THE FOIL SEAL