

OSOM[®]

Mono Test



TO AID IN THE DETECTION OF MONONUCLEOSIS

FOR USE WITH WHOLE BLOOD, SERUM, OR PLASMA

FEATURES:

- ✓ **ACCURATE;** >99% sensitivity; 96% specificity
- ✓ **FAST;** Results in 5 minutes
- ✓ **EASY-TO-READ;** Two-color results
- ✓ **WORLD-CLASS TECHNICAL SUPPORT;** Staffed with certified medical technologists and skilled professionals



TEST PROCEDURE / RESULTS



For serum, plasma or whole blood from collection tubes, add 1 drop to test tube from pipette.

For fingertip blood, fill capillary tube and dispense blood into test tube.



Add 1 drop of diluent to test tube. Mix.


Place the test stick into the treated sample.



5 MIN



Read results at 5 minutes.

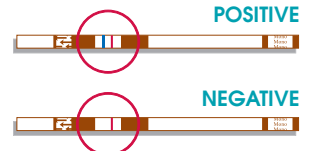

ORDERING INFORMATION

○ OSOM[®] MONO TEST

CONFIGURATION

25 Tests

CATALOG NUMBER

145 OR 145E (CE marked)



Distributed by: **CLIAwaived.com[™]**
 San Diego, CA 92121
 tel 858-481-5031
 toll free 888-882-7739
 www.cliawaived.com

