

iScreen™ OFD



Oral Fluid Drug Screen

Corporate Drug Testing Made Simple.

- A simple mouth and gum swab with built-in test
- Four, five and six panel drug configurations offered
- Photocopy template included for recording results
- Easy to read results - no machine required

Brands You Can Trust™



iScreen™ OFD

Oral Fluid Drug Screen Corporate Drug Testing Made Simple.

Distributed by: **CLIAwaived.com™**
San Diego, CA 92121
tel 858-481-5031
toll free 888-882-7739
www.cliawaived.com



iScreen™ OFD
6-panel shown as
a negative result.

| Drug Name: | Abbr: | Cutoff: |
|----------------------|-----------|---------|
| Amphetamine..... | AMP..... | 50ng/ml |
| Methamphetamine..... | mAMP..... | 50ng/ml |
| Marijuana..... | THC..... | 12ng/ml |
| Phencyclidine..... | PCP..... | 10ng/ml |
| Cocaine..... | COC..... | 20ng/ml |
| Opiates..... | OPI..... | 40ng/ml |


Configurations:

4-panel...I-DSB-745-021.....COC, THC, OPI, mAMP

5-panel...I-DSB-755-011.....COC, THC, OPI, mAMP, AMP


6-panel...I-DSB-765-011.....COC, THC, OPI, mAMP, AMP, PCP

Instructions

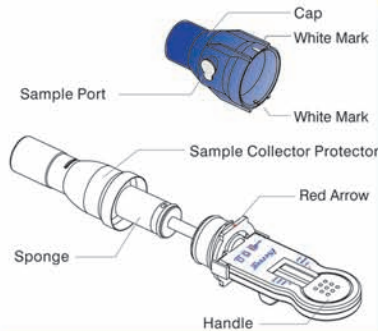


Section I: Sample Collection

1. Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
2. Remove Sample Collector Protector.
3. Insert Sponge end of collector into mouth. Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure **complete saturation**. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing Sponge. Hand the saturated collector over to the tester.

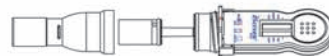


Section II: Testing

1. A. Align the Red Arrow on the device with either one of the White Marks on the Cap. Insert the collector vertically into the Cap and **press down firmly until the Cap reaches the thread**.
B. Twist the Handle clockwise 180° to tighten the Cap until the Red Arrow lines up with the other White Mark.
C. Place the test device horizontally on a clean and level surface with facing up.

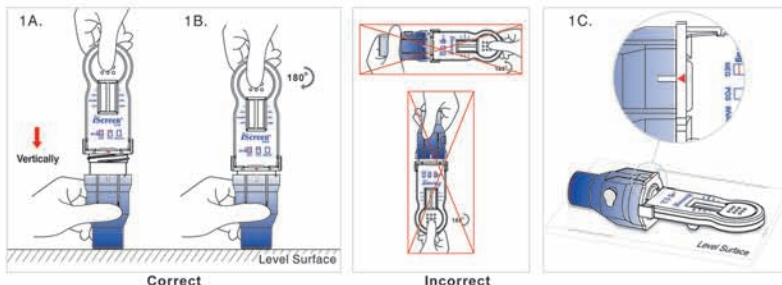

2. Read results at ten (10) minutes. Results stable up to sixty (60) minutes.

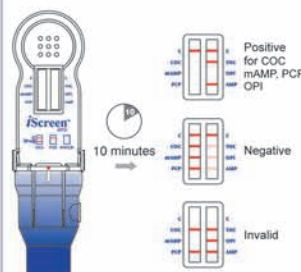

* All positive results are presumptive and should be confirmed by an alternate method. The preferred confirmatory method is GC/MS or GC/MS/MS.

3. If the oral screen registers a "non-negative" result, **discard product**, hand the donor a Custody and Control Form and ask them to proceed to your assigned collection facility for a urine drug screen. Advise the donor to go directly to the collection site.