

iScreen™ Mono

Distributed by: CLIAwaived.com™
San Diego, CA 92121
tel 858-481-5031
toll free 888-882-7739
www.cliawaived.com



CLIAwaived.com™



Mononucleosis Rapid Test Cassette

Simple, point-of-care detection of Infectious Mononucleosis

- CLIA Waived whole blood procedure
- On board procedural control
- Room temperature storage
- Results in minutes

Brands You Can Trust™



iScreen™ Mono

Mononucleosis Rapid Test Cassette
Simple, point-of-care detection of Infectious Mononucleosis



Kit includes:

- External positive/negative controls
- Developer buffer
- Transfer pipette



Features

Screens whole blood,
serum & plasma samples.....
CLIA Waived for

Benefits

whole blood samples..... Simple laboratory procedure
Results in minutes..... Rapid results for early diagnosis
Room temperature storage..... Convenient handling
Very high overall accuracy..... Confidence in test results

Product Code:	Kit Size:	Description:	Complexity:
I-MO-402.....	20/BX.....	Cassette.....	CLIA Waived/whole blood, Moderate/serum, plasma

iScreen™ Mono positive result shown above.

Instructions for I-MO-402

Procedure

Read procedure before running test. Refer to package insert for details.

Place test device on a clean, dry, level surface.

Venipuncture Whole Blood


Add 2 Drops of Blood (about 50 µL), then 1 Drop of Buffer to the Sample Well.

Fingerstick Whole Blood

Add 1 Capillary Tube of Blood (about 50 µL), then 1 Drop of Buffer to the Sample Well.

Serum/Plasma

Add 1 Drop of Serum or Plasma (about 25 µL), then 1 Drop of Buffer to the Sample Well.

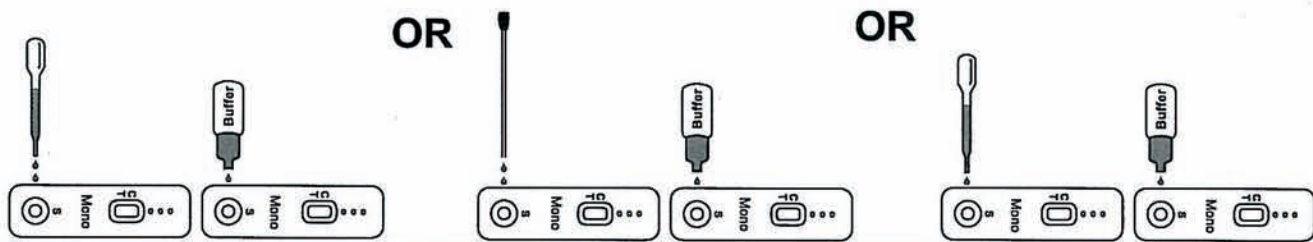

Interpretation of Results



Read result at five (5) minutes. Do not move test device until ready to read.



Any shade of red in the test line region (T) should be considered positive.