



CLEARVIEW™

C. DIFF A

Highly accurate results right before your eyes



*For the qualitative detection of
Clostridium difficile toxin A from fecal samples.*

One-Step Procedure

Simple Sample Preparation

Proven Performance

Rapid Turn Around



Wampole
Laboratories™

CLEARVIEW™ C. DIFF A

Performance Characteristics

Six hundred forty-two patients with antibiotic-associated diarrhea or risk for nosocomial infection were evaluated by Clearview™ C. DIFF A, cell cytotoxicity and specific culture for *C. difficile*. There was 92.2% correlation between Clearview C. DIFF A and the reference method (cell toxicity with neutralization).

		Positive	Negative	Total
Cell Cytotoxicity	Positive	123	30	153
	Negative	20	469	489
	Total	143	499	
Relative Sensitivity:	80.4%			
Relative Specificity:	95.9%			
Overall Agreement:	92.2%			



One-Step Procedure

- Patented format eliminates the need for multi-step procedures. Sample addition is the only test procedure step.

Simple Sample Preparation

- One-step Wampole Stool Filtration System makes sample preparation simple and clean.

Proven Performance

- Monoclonal antibody technology provides results with 80.4% relative sensitivity and 95.9% relative specificity.

Rapid Turn Around

- Results are available just 30 minutes after sample addition.

Reimbursement Information


CPT Code 83516

Immunoassay for analyte other than antibody or infectious agent antigen, qualitative or semiquantitative; multiple step method.

Simple test procedure and results interpretation*

Prepare Sample:

- Use the Wampole Stool Filtration System, or
- Centrifuge with a bench top centrifuge, 10 min. at 13,500 rpm.

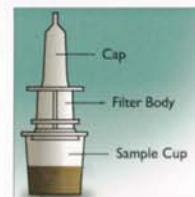


Add Sample:

- Add six drops/125µL of filtrate to Sample Window.



Read Results:

- Interpret results in 30 minutes.
- A blue line in the Results Window indicates a positive result.
- An internal control feature will show a blue line in the Control Window if the test has been performed correctly.

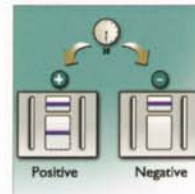

* Refer to package insert for complete instructions for use.

Order Information

Product Code

Clearview™ C. DIFF A 20 test kit	135050
Clearview™ C. DIFF A Stool Filtration Devices 20 units	135051
Also Available	
Clearview™ Strep A 20 test kit	135007
Clearview™ Chlamydia 20 test kit	135301
Clearview™ HCG II Urine - 20 test kit	135002
Urine - 100 test kit	135006
Clearview Easy hCG 20 tests	135010

Authorized Distributor



Wampole Laboratories, PreVue and the Wampole Laboratories logo are a registered trademark of Carter-Wallace, Inc.

Wampole Laboratories
Division of Carter-Wallace, Inc.
Cranbury, NJ 08512-0181

800-257-9525 Telephone
800-532-0295 Facsimile
www.wampolelabs.com Internet