



Respiratory | Women's Health & STD | Enterics & GI | General Health | Instrumentation

Clearview

RUBELLA-plus

For the qualitative detection of Rubella antibody (IgG) in human serum or plasma.



RUBELLA-plus[®]

Clearly Different

Wampole Rubella-plus provides simple, accurate detection of rubella antibodies.

- **Fast: results available in just 15 minutes**
- **Easy: no special training or rotator needed**
- **Reliable: detects antibodies at a level greater than or equal to 10 IU/mL**
- **Flexible: can be used with either serum or plasma**

Clearly better

The rapid, reliable results provided by Rubella-plus can lead to improved patient care.

- **Early detection in women of child-bearing age may prevent the transmission of the disease to unborn children**

Clearview
RUBELLA-plus
 30 Test Kit
 Order No. 84R2
 CLIA Non-waived


About Rubella

Rubella (also called German measles or 3-day measles) is a disease caused by the rubella virus. Rubella spreads from person to person through the air (often through close contact such as talking, coughing, or sneezing) and is moderately contagious.


Although rubella can strike people of all ages, **it poses the greatest danger to unborn babies.** Congenital rubella syndrome (CRS) occurs when the rubella virus attacks a developing fetus. Up to 85% of infants infected during the first trimester will be born with birth defects, including deafness, blindness, heart defects, and mental retardation.¹ Miscarriages are also common.²

Fortunately, due to widespread immunization with a rubella vaccine, rubella and CRS are now uncommon in the U.S.¹ Rubella vaccine is often given together with measles and mumps vaccines as part of an MMR shot.

RUBELLA-plus Test Procedure

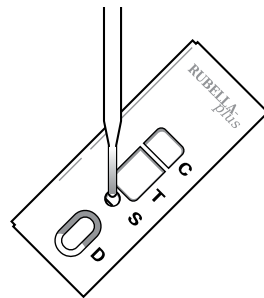


Add 10µL of serum or plasma to the Sample well (S)

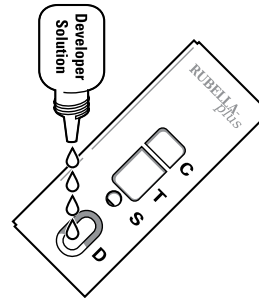

Follow with 3 to 4 drops Developer Solution (D)



Allow 15 minutes

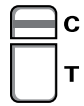

1 Band = Negative



2 Bands = Positive

Interpret results

1. Immunization Action Coalition. Vaccine Information for the Public and Health Professionals: Rubella Questions and Answers. Available at: <http://www.vaccineinformation.org/rubella/qandadis.asp>. Accessed April 21, 2005.
2. Centers for Disease Control. Parents Guide to Childhood Immunization: Rubella (German Measles). Available at: <http://www.cdc.gov/nip/publications/Parents-Guide/pg-rubella.pdf>. Accessed April 21, 2005.

RUBELLA-plus® is a registered trademark of the Inverness Medical family of companies.

331 S. 104th Street
 Louisville, CO 80027 USA
www.invernessmedicalpd.com

1.800.637.3717
 P: 303.530.3888
 F: 800.637.3827

© 2006 Inverness Medical - BioStar Inc.
 02831.00

visit
www.invernessmedicalpd.com
 for more information