


We made a well so big, it's hard to miss.

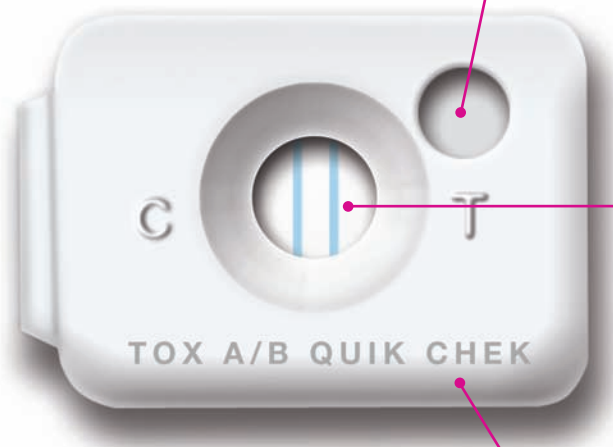

EASY TO USE

- A larger sample well means neater, quicker placement
- Novel patent-pending technology smoothly disperses sample within the device
- The chance of clogging is minimized, greatly reducing unwanted spillovers

EASY TO READ

- Two lines mean *C. difficile* toxin A/B is present
- Internal control line validates the QC result in the same reaction window

HIGHLY ACCURATE

- >90% sensitivity*
- >99% specificity*

TOX A/B QUIK CHEK™ has a larger sample well to make testing for *C. difficile* toxins A and B fast, easy, accurate, and clean.

CLIAwaived.com™

For more information please contact CLIAwaived Inc. at
11578 Sorrento Valley Road ■ San Diego, CA 92121

Tel: (858) 481-5031 ■ Toll Free: (888) 882-7739

E-mail: info@cliawaived.com

Visit us on the web at: www.cliawaived.com

Distributed by: CLIAwaived, Inc.

TOX A/B QUIK CHEK™ from Inverness Medical

A rapid IVD test for detecting *C.difficile* toxins A and B in fecal specimens



Cat. #30394, 25-test kit

Everything you need...

- 25 membrane device pouches, each containing a device and dessicant
- Diluent (14 mL)—buffered protein solution furnished with graduated dropper assembly
- Wash buffer (10 mL)—a buffered solution supplied with graduated dropper assembly
- Conjugate (2 mL)—HRP-labeled mouse MAB specific for toxin A and HRP-labeled goat polyclonal antibody specific for toxin B
- Substrate (3.5 mL)—TMB solution
- Positive control (1 mL)—antigen in buffered solution
- Disposable plastic transfer pipettes—50

... to run this simple, rapid test†

1. Add one drop of conjugate to 500 μ L diluent-mix.
2. Add 25 μ L of specimen to conjugate/diluent mixture, transfer 400 μ L of final dilution mixture to sample well. Incubate at room temperature for 15 minutes.
3. Add 300 μ L wash to reaction window, allowing it to completely absorb.
4. Add 2 drops substrate to reaction window.
5. Incubate at room temperature for 10 minutes. Read results.

CLIAwaived.com

For more information please contact CLIAwaived Inc. at
11578 Sorrento Valley Road ■ San Diego, CA 92121
Tel: (858) 481-5031 ■ Toll Free: (888) 882-7739

E-mail: info@cliawaived.com

Visit us on the web at: www.cliawaived.com

Distributed by: CLIAwaived, Inc.