

# ACON Strep A Rapid Test Strip

Distributed by

CLIAwaived™ Inc.



The ACON Strep A Rapid Test Strip is a rapid chromatographic immunoassay for the qualitative detection of Strep A antigen from throat swab specimens to aid in the diagnosis of Group A Streptococcal infection. Providing results within 5 minutes, the test utilizes antibodies specific for whole cell Lancefield Group A Streptococcus to selectively detect Strep A antigen in a throat swab specimen.

#### **BENEFITS:**

- Variety of formats for flexible testing
- On board procedural control
- High sensitivity and specificity
- Room temperature storage
- Results in 5 minutes
- FDA-Cleared / CLIA Waived

#### **Ordering Information:**

Product Description	Catalog #
ACON Strep A Rapid Test Strip (25 tests)	IST-502T

#### **Reimbursement Information:**

CPT Code #	National Limit Amount
87880QW – Rapidly detects GAS antigen from throat swabs and used as an aid in the diagnosis of GAS infection, which typically causes strep throat, tonsillitis, and scarlet fever.	\$17.08

DISCLAIMER – The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for health care services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived™ Inc. All CPT codes are supplied for information purposes only and represent no statement; promise or guarantee by CLIAwaived™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Associations. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercial payors. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

# ACON Strep A Rapid Test Strip Instructions

**Sensitivity 90%**  
**Specificity 94%**

- Kit components  
 25 Test devices  
 25 Swabs  
 1 Procedure card  
 1 Reagent A  
 1 Reagent B  
 1 Positive control  
 1 Negative control

## I-ST-502T (Strep-A One Step - Waived)

1. Holding bottles upright, add 5 drops of Reagent A, then add 5 drops of Reagent B to Chamber.

2. Add Swab to Chamber. Agitate Swab vigorously in the solution 10 times. Let stand 1 minute.

3. To Remove Swab:  
 • Hold down Chamber with thumb and index finger.  
 • Lift Swab halfway up the Chamber and press against the ribs inside the wall of the Chamber.  
 • Rotate Swab 5 times while pressing firmly against ribs to extract liquid.  
 • Discard Swab.

4. Hold down device on a flat surface. Open Valve by twisting ribbed part of Valve clockwise until it stops (Open Valve Position). Begin timer. Read results at five (5) minutes.



### I-ST-501 Interpretation of Results



### I-ST-502 Interpretation of Results



### I-ST-502T Interpretation of Results



\*Any shade of red in the test region (T) should be considered positive. \*Any shade of red in the test region (T) should be considered positive. \*Any shade of red in the test region (T) should be considered positive.