

# Hgb Pro


Professional Hemoglobin Testing System®

CLIAwaived.com™

CLIAwaived.com - 4332 Corte de la Fonda - San Diego, CA 92130

Toll Free : (888) 882-7739 - Phone (858) 481-5031

FAX: (801) 720-7568 - www.cliawaived.com

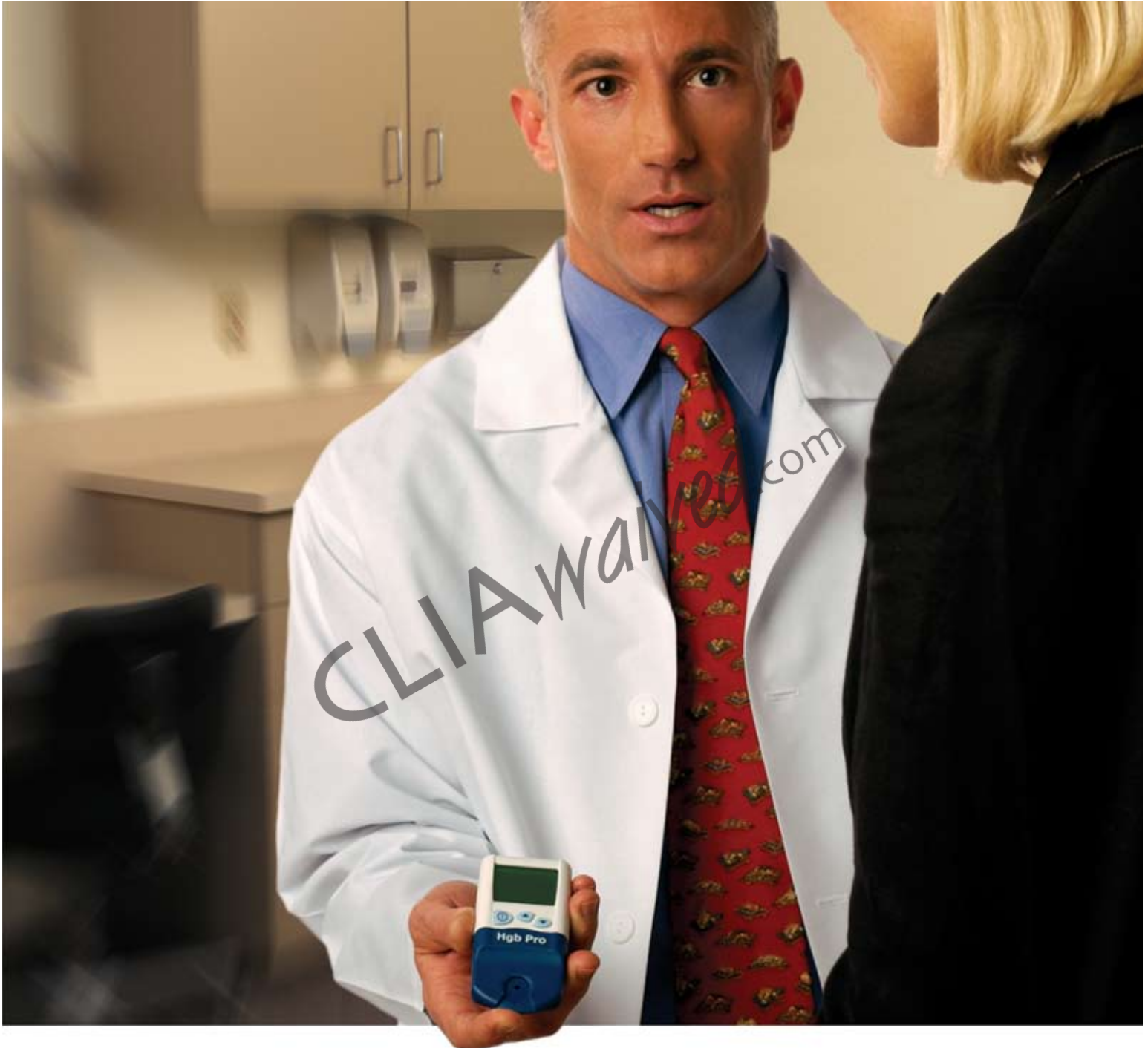

*Precision and accuracy can be affordable.*

*Trust the point-of-care industry leaders.*

## SIMPLE TO USE

The Hgb Pro Professional Hemoglobin Testing System is portable and easy to use. Simply insert a test strip into the meter and place a small drop of blood onto the strip. Since the Hgb Pro is self-calibrating, no additional calibration is required.

## FEATURES

- 250 memory data storage
- Large, easy-to-read LCD screen
- Operates on one 3-volt lithium battery
- Cost-effective

- CLIA-waived
- Small blood-drop sample
- Results within 30 seconds
- Lab-quality accuracy and precision

## SYSTEM OVERVIEW

### THREE SIMPLE STEPS

Step 1: Turn on meter.

Step 2: Insert test strip.

Step 3: Apply blood.

### QUALITY CONTROL

Liquid quality controls, Para 4<sup>®</sup> (product number 215110), are available from Streck<sup>®</sup> Laboratories (1-800-843-0912 or [www.streck.com](http://www.streck.com)) for the Hgb Pro. It is recommended that quality control testing be performed upon receipt of a new lot of test strips and once every three months thereafter.

### ORDERING INFORMATION

HGBPROKIT: Starter kit (Contains one meter and one bottle of 25 test strips.)

HGBMETER: One Hgb Pro meter

HGBTEST100: Test strips (Contains four bottles of 25 test strips.)

### REIMBURSEMENT INFORMATION

Use CPT code 85018QW.



Actual Size

**CLIA**waived.com<sup>TM</sup>

CLIAwaived.com - 4332 Corte de la Fonda - San Diego, CA 92130  
Toll Free : (888) 882-7739 - Phone (858) 481-5031  
FAX: (801) 720-7568 - [www.cliawaived.com](http://www.cliawaived.com)