

Clinically Proven, Safe and Easy-to-Use



Histofreezer®

Portable Cryosurgical System

For more information, visit www.Histofreezer.com

New Product Configurations

60M2C — ECONOMY KIT

- 60-120 applications per kit*
- (24) 2mm and (36) 5mm applicators
- Up to three-year shelf-life
- Ideal for small and large lesions

36F1C — 5MM VALUE KIT

- 36-72 applications per kit*
- (36) 5mm applicators
- Up to three-year shelf-life
- Ideal for treatment of larger lesions and plantar warts

36M1C — VALUE KIT

- 36-72 applications per kit*
- (12) 2mm and (24) 5mm applicators
- Up to three-year shelf-life
- Ideal for small and large lesions

40T1C — 2MM VALUE KIT

- 40-80 applications per kit*
- (40) 2mm applicators
- Up to three-year shelf-life
- Ideal for treatment of smaller lesions

*See Directions for Use for more information

Reimbursement

Histofreezer is covered by most private and Government insurance plans and offers high reimbursement rates.⁴

National Medicare Average Reimbursement Rates

Indication	CPT Code	Amount
Acrochordon (Skin Tags)	11200, 11201	\$18-\$66
Actinic Keratosis	17000, 17003, 17004	\$74-\$162
Common Warts	17110, 17111	\$101-\$120
Genital Warts	46916, 46924, 54056, 54065, 56501, 56515	\$125-\$465
Lentigo	17110, 17111	\$101-\$120
Molluscum	17110, 17111	\$101-\$120
Molluscum Contagiosum	46916, 46924, 54056, 54065, 56501, 56515	\$125-\$465
Seborrheic Keratosis	17110, 17111	\$101-\$120

For a list of authorized distributors visit www.Histofreezer.com

References:

1. Data on file, OraSure Technologies, Inc., Bethlehem, PA.
2. F. Caballero, Martinez, MD, et al. Dermatological cryosurgery in primary care with dimethyl ether propane spray in comparison with liquid nitrogen. Translated from: *Atercion Primaria*. 1996; Vol. 18 No. 5 (211, 2216).
3. Histofreezer Portable Cryosurgical System Package Insert-US, OraSure Technologies, Inc., Bethlehem, PA.
4. Histofreezer Portable Cryosurgical System Getting Reimbursed for Cryosurgery [Brochure]. Bethlehem, PA: OraSure Technologies, Inc.; 2010.
5. Histofreezer Portable Cryosurgical System Product Detailer [Sales Sheet]. Bethlehem, PA: OraSure Technologies, Inc.; 2009.
6. Study on AK & Lentigo, Claire Mills, RN, Sonora Women's Clinic, 1999.
7. St. Luke's Hospital Study on Condyloma, Thomas Gaydos, MD, et al, 1996.
8. Study on Plantar Warts, James Ricketti, DPM, Hightstown, NJ, 1992.
9. Study on the effectiveness of Dimethyl Ether and Propane, W.J.B. Swart, MD, 1993.



Distributed by: **CLIAwaived.com™**
 San Diego, CA 92121
 tel 858-481-5031
 toll free 888-882-7739
www.cliawaived.com

A Cool Way to Remove Common Warts & Benign Lesions



"Histofreezer is an excellent solution for removing children's warts. The treatment is short and painless. I also like the fact that Histofreezer doesn't contain CFC."

John Niziol, MD
 Pediatrician, Wayne, NJ



For more information, visit www.Histofreezer.com

Histofreezer® Portable Cryosurgical System

Clinically Proven, Safe and Easy-to-Use



Histofreezer®

Portable Cryosurgical System

"I have been using Histofreezer for years. It's easy-to-use, works well, and my patients really like it."

Edwin S. Hart III, DPM
Podiatrist, Bethlehem, PA


**NO
HAZMAT
FEE**

Histofreezer is an ultra-portable, patient friendly and easy-to-use cryosurgical system with proven efficacy and safety.¹

Histofreezer


- Clinically proven, safe and easy-to-use^{1,3}
- Results are comparable to Liquid Nitrogen²
- Histofreezer is patient friendly with a low risk of pain and scarring³
- Application takes 40 seconds or less³
- Does not require a Freeze-Thaw-Freeze procedure³
- Does not require the use of messy and leaky cones, unlike other cryosurgical products³
- Requires no bulky equipment or initial capital investment, unlike Liquid Nitrogen
- Histofreezer is CFC-Free and ozone-friendly³
- Economically priced and reimbursable^{4,5}
- **NO HAZMAT FEE** + All Inclusive Kits = Greater ROI

Histofreezer vs. Liquid Nitrogen²

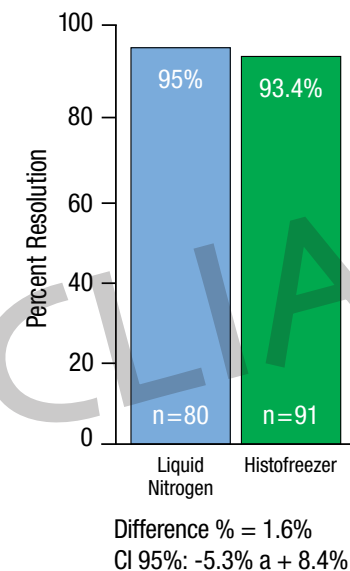

Efficacy

- The efficacy of Histofreezer has been proven through clinical studies and clinical use spanning over 19 years^{2, 6-9}
- Easier, faster and less invasive than cutting or burning

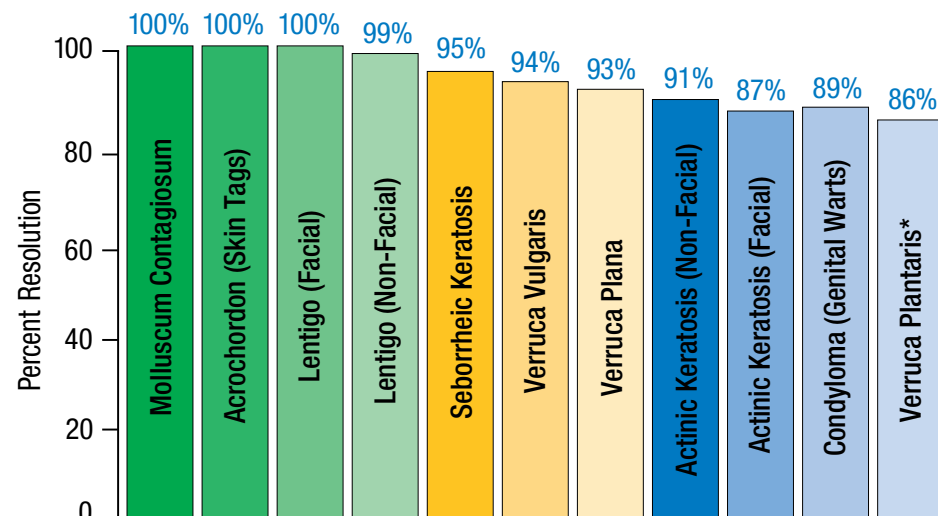

*Histofreezer with post treatment of 27% salicylic acid/17% lactic acid.

86%-100% resolution rates for warts and benign lesions.

Indications and Treatment Times

• Acrochordon (Skin Tags)	40 seconds	• Molluscum Contagiosum	20 seconds
• Actinic Keratosis (Facial)	15 seconds	• Seborrheic Keratosis	40 seconds
• Actinic Keratosis (Non-Facial)	40 seconds	• Verruca Plana (Flat Warts)	20 seconds
• Condyloma Acuminata (Genital Warts)	40 seconds	• Verruca Plantaris (Plantar Warts)	40 seconds
• Lentigo (Facial)	15 seconds	• Verruca Vulgaris (Common Warts)	40 seconds
• Lentigo (Non-Facial)	40 seconds		

Safety

Histofreezer is safe for use on adults and children 4 and over.³ Selecting either the 2mm or the 5mm Histofreezer applicator allows targeted treatment of the wart or benign lesion reducing the chance of scarring.²

Patient Friendly and Easy as 1-2-3

Histofreezer is user-friendly and requires just one hand to perform the procedure. Histofreezer ensures better patient compliance and the usage is easy as 1-2-3. Treatment times are listed on the canister, no guessing!



1 CHARGE

Charge the applicator by pressing valve until droplets start to form.



2 INVERT

Invert the canister for 15 seconds to achieve optimal freezing temperature.



3 APPLY

Apply the frozen applicator to lesion for 15-40 seconds. See *Histofreezer Directions for Use* for indications and treatment times.

IMPORTANT: DO NOT PRESS VALVE WHILE THE HISTOFREEZER APPLICATOR IS IN CONTACT WITH PATIENT'S SKIN.³

Comparison of Treatment Methods

Histofreezer vs. Liquid Nitrogen

A clinical study comparing Histofreezer with Liquid Nitrogen demonstrated comparable results for the treatment of warts and benign lesions.² In addition, Histofreezer offers portability and ease-of-use in a disposable format and no tank refills or dedicated treatment rooms are needed.

Histofreezer vs. Other Portable Cryosurgical Products

Compared to other products, Histofreezer can be less expensive per treatment visit; it does not require a Freeze-Thaw-Freeze procedure or messy and leaky cones, and it is also ozone-friendly.³

Histofreezer vs. Acids (TCA) or Other Topicals

Acids and topicals can be painful, offer lower resolution rates and may require self-treatment by patients, therefore causing a loss of revenue for the practice.³

Histofreezer®

Portable Cryosurgical System