


Add Extraction Buffer

Fill the dropper to the line marked on the barrel and expel entire contents into tube.




Add Swab To Buffer

Add swab to the tube with the extraction buffer.

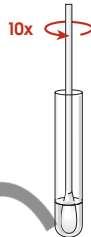

Mix

Vigorously rotate and twist the swab against the side of the tube at least 10 times.

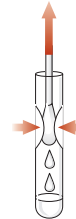

Squeeze Liquid From Swab

Squeeze the sides of the tube to express as much liquid as possible. Dispose of swab properly.



Read results at 10 minutes

Add Test Stick

Place test stick (arrows pointing downward) into the tube with buffer. Read results at 10 minutes.

Reading Test Results

POSITIVE RESULTS

NOTE: A pink-to-purple line of *any intensity or thickness* in the A or B region is considered a positive result.

LINE PLACEMENT GUIDE



INFLUENZA A POSITIVE

One line in the control line position, and one line in the "A" test line position.



INFLUENZA B POSITIVE

One line in the control line position, and one line in the "B" test line position.

Note: It is possible to have 3 lines, which would indicate a positive test for Influenza A and Influenza B.



NEGATIVE RESULTS

One line in the control line position, and no lines at either the "A" or the "B" test line positions.



INVALID RESULTS

No line appears at the control line position. Repeat the test using a new sample and a new test dipstick.

