

QUICKVUE[®]

Influenza A+B TEST

Test and Treat Today!

Diagnosing influenza can be difficult. One third of all upper respiratory infections are caused by influenza. This leads to approximately 36,000 deaths, over 200,000 hospitalizations¹ and accounts for \$10-\$15 billion spent in lost work and related medical expenses each year in the U.S.²

Quidel's CLIA-waived QuickVue[®] Influenza A+B test provides you with differential A or B test results allowing you to test and treat in just one visit. Prompt type-specific diagnosis enables a choice of antiviral therapy, helps avoid the overuse of antibiotics, and prevents healthcare costs related to unnecessary testing and treatment.³



The QuickVue Influenza A+B test has also been shown to detect cultured avian influenza viruses, including avian Influenza A subtype H5N1 virus; however, as with other rapid tests for influenza, the ability of the QuickVue Influenza A+B test to detect influenza Type A in patients infected with H5N1 has not been established.

The QuickVue Influenza A+B test is CLIA waived, sensitive and specific, giving you the results you need. Right. Now.

¹<http://www.cdc.gov/flu/keyfacts.htm>

²Rothberg, M.B. Management of Influenza Symptoms in Healthy Adults, J. Gen Intern Med 2003; 18:808-815.

³Montalto, N.J., D.O. An Office-Based Approach to Influenza: Clinical Diagnosis and Laboratory Testing, Am Fam Phys 2003; 67(1):111-118.

Rapid Diagnosis and Treatment

- Identifies influenza Type A or B in positive samples, in about 10 minutes, for better patient management decisions.
- Most patients tested can be diagnosed and treated in just one visit.

Accurate Detection

- QuickVue[®] Influenza A+B test detects both types A and B viral antigens.
- Accurate detection with nasal swab samples: 94% sensitivity and 90% specificity for type A 70% sensitivity and 97% specificity for type B
- See reverse side for additional performance claims.

Easy to Use

- 3 steps
 - 1 reagent, 90 seconds hands-on time.
- 1 test procedure provides results for both A and B.

Convenient Packaging


- Each kit contains everything needed to perform the test, including foam-tipped nasal swabs and Positive and Negative Controls.
- Available in 25-test kit.
- Room temperature storage with 24-month shelf life from date of manufacture.
- Transport swab set also available separately.

QUIDEL[®]
CORPORATION

For more information on QuickVue[®] Influenza A+B test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • www.quidel.com

Nasal/Nasopharyngeal swab procedure:

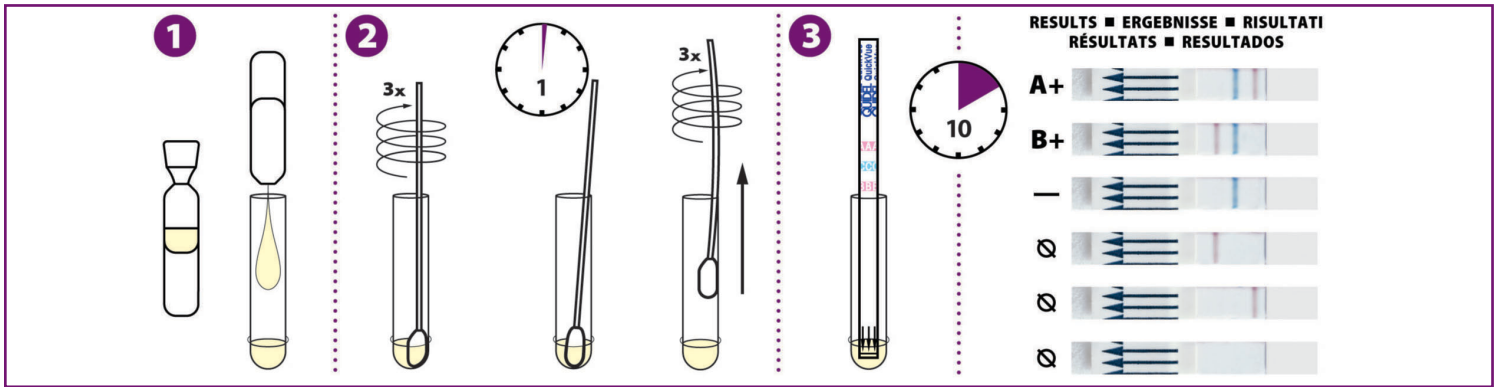

Nasal Swab:

Sensitivity: A – 94%, B – 70%
Specificity: A – 90%, B – 97%

Accuracy: A – 91%, B – 93%

PV (+): A – 62%, B – 82%

PV (-): A – 99%, B – 94%

Nasopharyngeal Swab:


Sensitivity: A – 83%, B – 62%
Specificity: A – 89%, B – 98%

Accuracy: A – 88%, B – 94%

PV (+): A – 67%, B – 80%

PV (-): A – 95%, B – 95%

Nasal wash/aspirate procedure:

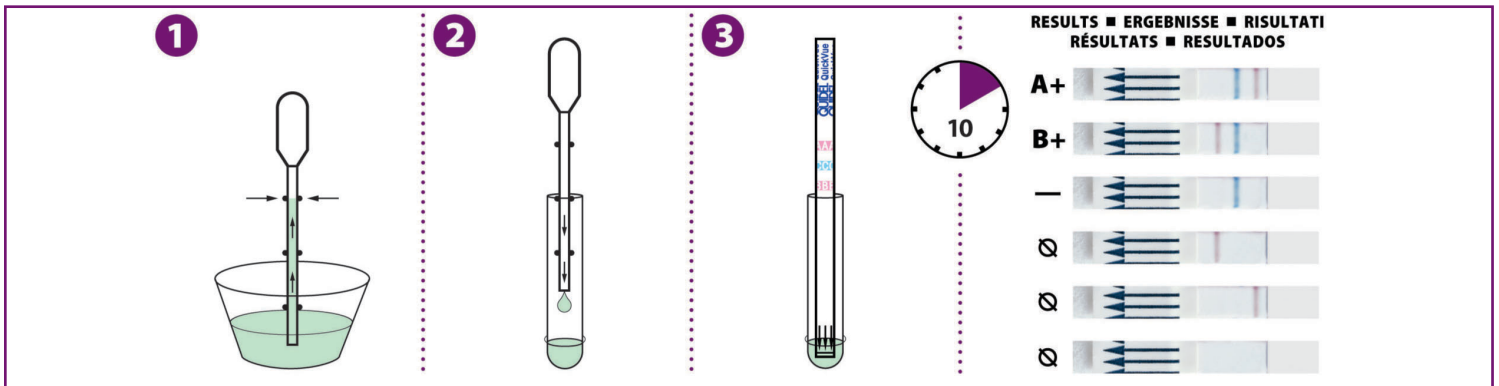

Fresh nasal wash/aspirate:

Sensitivity: A – 77%, B – 82%
Specificity: 99%

Accuracy: A – 95%, B – 96%

PV (+): A – 91%, B – 90%

PV (-): A – 96%, B – 97%

Frozen nasal wash:

Sensitivity: A – 86%
Specificity: A – 95%

Accuracy: A – 91%

PV (+): A – 93%

PV (-): A – 90%

The performance of any rapid flu test is dependent on sample collection and handling and the adherence to the Package Insert. Refer to our website at www.quidel.com for additional performance claims.



To order: Call your Quidel Account Manager or your local distributor

QuickVue® Influenza A+B – 25 Test Kit:
Catalog #20183



QuickVue® Swab Transport Tube Set
Catalog #20171



For more information on QuickVue® Influenza A+B test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • www.quidel.com

Distributor Information: