

5.6.1 Connecting a printer

The **HemoPoint® H2** photometer supports the connection of a Seiko Model No. DPU-414 serial printer.



Illustration 32: DPU-414 printer.

The DPU-414 can be ordered through an authorized Stanbio Laboratory distributor.

With the use of a printer you can print each patient's test result immediately or print patient result(s) from memory (data storage) later. See Section 5.2.2 "Printing out results".

5.6.1.1 Connecting the printer cable

To connect the printer to the **HemoPoint® H2** a special printer cable is needed which is included with each printer if sold for use with the **Hemo Point® H2** or may be purchased separately from an authorized Stanbio Laboratory distributor.

Please only use the cable intended for this purpose, otherwise both the photometer and the printer may be seriously damaged.



WARNING!

The printer cable is equipped at one end with a 9-pin serial plug to connect with the printer and at the other end with a 6 pole mini –DIN plug to connect to the interface socket of the **HemoPoint® H2** (see Illustration 33, Connecting the Seiko DPU-414 printer).



Illustration 33: Connecting the Seiko DPU-414 printer.