

# OSOM<sup>®</sup>



## Strep A Test

*Clear, accurate results in just minutes can help everyone feel better before the office visit is over.*



### ✓ VERY ACCURATE

96% sensitivity; 98% specificity.

### ✓ FAST

Results in 5 minutes.

### ✓ EASY-TO-READ


Two-color results: blue test line, red control line.

### ✓ CONVENIENT


Compact packaging. Everything you need to run 50 tests. Kit Includes positive and negative controls. Room temperature storage.

### TEST PROCEDURE / RESULTS

Add 3 drops Reagent 1 and 3 drops Reagent 2 to the test tube.

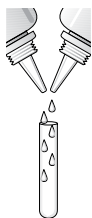

Insert swab and rotate 10 times vigorously.

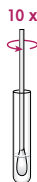


10 x

Let stand for 1 minute.



1 MIN


Express liquid from the swab by squeezing the sides of the tube as the swab is withdrawn.

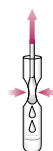

Place the test stick into the extracted sample.



5 MIN Read results at 5 minutes. Positive results may be read as soon as red control line appears.

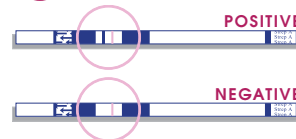

FOR MORE INFORMATION ON OUR RAPID TEST PRODUCTS, VISIT US AT [www.genzymediagnostics.com](http://www.genzymediagnostics.com)

genzyme  
Diagnostics

**U.S. Customers - To order, contact your medical supply distributor or call Genzyme at 800-332-1042**

#### ORDER:

#141, OSOM<sup>®</sup> Strep A Test, 50 Tests

#141E, OSOM<sup>®</sup> Strep A Test, 50 Tests (CE Marked)

#### The Americas

31 New York Avenue  
Framingham, MA 01701-9322, USA  
Customer/Technical Service: 800-332-1042  
E-mail: [usdiagcustomerservice@genzyme.com](mailto:usdiagcustomerservice@genzyme.com)

ISO 13485 Certified

#### International

50 Gibson Drive  
Kings Hill, West Malling  
Kent, ME19 4AF, UK  
Tel: 44 (0) 1732-220022  
Fax: 44 (0) 1732-220024  
E-mail: [ukdiagcustomerservice@genzyme.com](mailto:ukdiagcustomerservice@genzyme.com)

ISO 9001:2000, ISO 13485

