



## New, Premium Oral Fluid Drug Test

#AV-SALIVA-5124A



**Intro Price:**  
**\$6.99 per test\***

**Regular Price:**  
**\$7.99 per test**

*Buy 200 or more for  
\$5.99 per test!*

### Detects 12 Drugs

<b>ALCO 0.02%</b>	<b>FEN 10</b>
<b>AMP 25</b>	<b>MET 25</b>
<b>BZO 10</b>	<b>MTD 30</b>
<b>BAR 50</b>	<b>OPI 30</b>
<b>BUP 5</b>	<b>OXY 20</b>
<b>COC 15</b>	<b>THC 40</b>
	Parent THC

- ✓ **Simple, 6 step procedure**
- ✓ **Read at 10 minutes, results stable for 1 hour**
- ✓ **Includes saturation indicator to ensure enough fluid has been collected and not adulterated**

The Oral Fluid Drug Screen Test is a rapid test that can be performed without the use of an instrument. The test utilizes monoclonal antibodies to selectively detect elevated levels of specific drugs in human oral fluid.

*This test is Forensic Use Only - Not FDA Cleared or CLIA waived.  
\*Requires a minimum of 25 tests. Intro price expires 11/30/2019.*

To order, call 858.630.9944 or email [sales@cliawaived.com](mailto:sales@cliawaived.com)



# PREMIUM ORAL FLUID DRUG TESTING

## SIMPLE PROCEDURE, 10 MINUTE RESULTS



### Step 1

Unscrew the Collector Cap to pull out the collection stick with the sponge from the Collection Chamber.



### Step 3

Remove the sponge from the mouth. Place the collection stick with saturated sponge into Collection Chamber.



10 Minutes

### Step 5

Peel off the label to read test results.




### Step 2

Put collector stick between tongue and cheek to collect oral fluid. Swab the inside of the mouth and gently chew or agitate sponge until the saturation stick appears red indicating that sufficient fluid has been collected - about 3 minutes.



### Step 4

Secure the cap, shake three times, then start the timer.

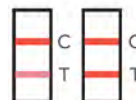

### Step 6

Read results at 10 minutes.  
Do not read results after 1 hour.