




# ImmunoCard STAT® STREPA


A simple, rapid test for detection of Group A Streptococcal Antigen.

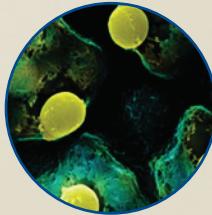

## Better patient care starts with STAT!

Group A Streptococcus is one of the most important causes of acute upper respiratory infection. Early diagnosis and treatment of Group A Streptococcus pharyngitis has been shown to reduce the severity of symptoms and further complications such as rheumatic fever.<sup>1</sup>

- 96% Sensitivity; 98% Specificity
- Results in just 5 minutes - shortening patient wait time
- The convenience of CLIA Waived, one-step technology, plus all required test materials conveniently packaged in each kit
- Easy-to-read: blue test line, red control line

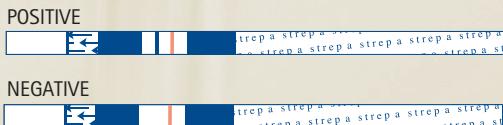


**Meridian  
Bioscience, Inc.**  
Inspired Science. Trusted Solutions.™


<sup>1</sup> Youmans, G.P., Paterson, P.Y. and Sommers, H.M. *Upper Respiratory Tract Infections: General Considerations, in the Biologic and Clinical Basis of Infectious Diseases*, W.B. Saunders Co, Philadelphia, 177-183, 1980.

## Test Procedure

**ImmunoCard STAT! STREP A**

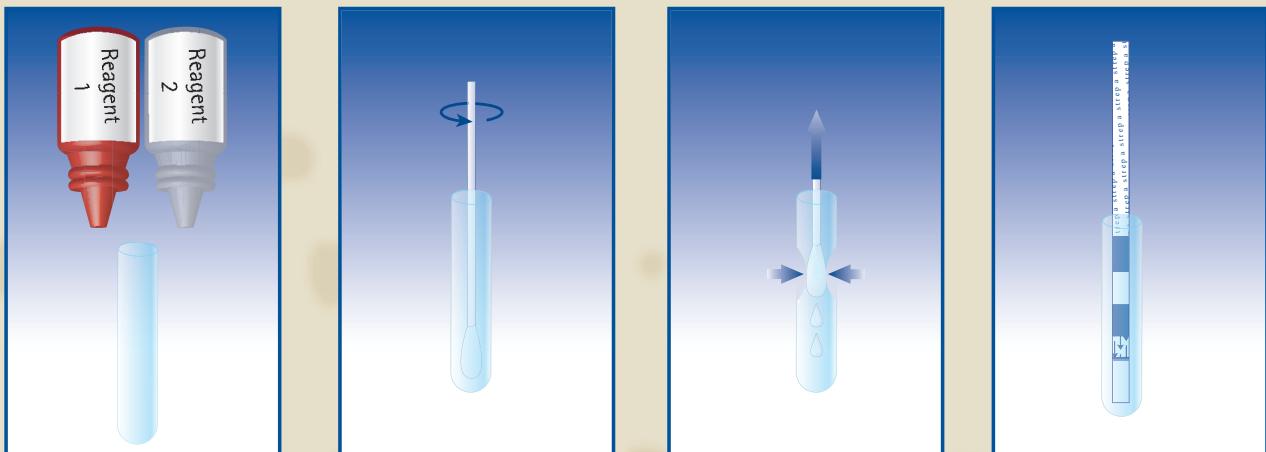

- Just before testing, add 3 drops of Reagent 1 and 3 drops of Reagent 2 to the Test Tube (the solution should turn light yellow).
- Immediately put the swab into the Tube. Vigorously mix the solution by rotating the swab forcefully against the side of the Tube at least 10 times. Let stand for 1 minute.
- Express as much liquid as possible from the swab by squeezing the sides of the Tube as the swab is withdrawn. Discard the swab.
- Place the absorbent end of the Test Stick into the extracted sample. Read results at 5 minutes.

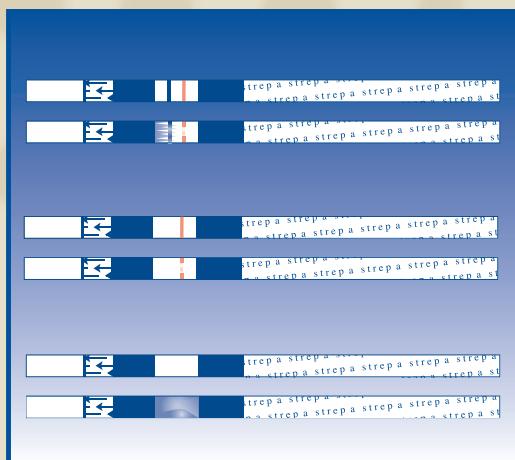

\*This chart does not contain complete instructions for use. For further information, please refer to the package insert.

## Ordering Information

*ImmunoCard STAT!® Strep A*

Test Size	Meridian#	Cardinal#	Fisher#
50 Test Strips	755250	B1041-76	23030153

**USA/Corporate Office:**  
3471 River Hills Drive  
Cincinnati, Ohio 45244  
Tel: 513.271.3700  
Orders/Cust Svc: 1.800.543.1980  
Tech Support: 1.800.343.3858  
Info Fax: 513.272.5432  
Order Fax: 513.271.0124  
[www.meridianbioscience.com](http://www.meridianbioscience.com)

**Europe b.v.:**  
Halderheiweg, 6  
5282 SN Boxtel  
The Netherlands  
Tel: +31 (411) 62 11 66  
Fax: +31 (411) 62 48 41  
[www.mdeur.com](http://www.mdeur.com)

**Europe s.a./n.v.:**  
Rue de l'Industrie, 7  
1400 Nivelles - Belgium  
Tel: +32 (67) 89 59 59  
Fax: +32 (67) 89 59 58  
[www.mdeur.com](http://www.mdeur.com)

**Europe France:**  
Le Quadra  
455, Promenade des Anglais  
06299 Nice Cedex 3 - France  
Tel: +33 (4) 93 18 72 10  
Fax: +33 (4) 93 18 72 11  
[www.mdeur.com](http://www.mdeur.com)

**Europe S.r.l.:**  
Via dell'Industria, 7  
20020 Villa Cortese (MI) - Italy  
Tel: +39 (0331) 43 36 36  
Fax: +39 (0331) 43 36 16  
[www.mdeur.com](http://www.mdeur.com)