



T-CUP

PROCEDURE CARD


1 Donor removes cap.

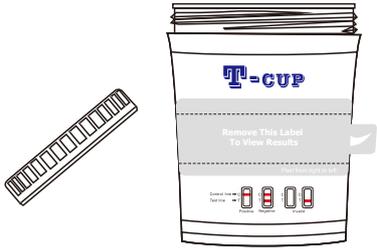


2 Donor provides specimen. Recap the cup and place the urine test cup on a flat surface.

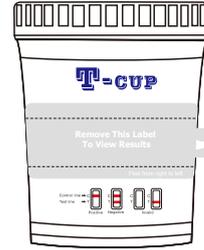

3 Technician peels off label to view results.

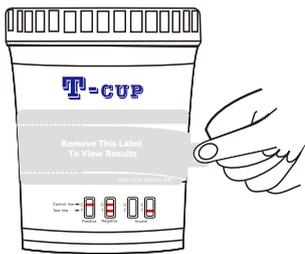


4 Read adulteration test results between 1-2 minutes.

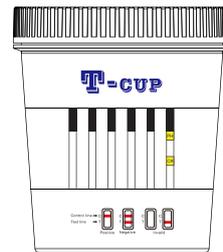


Note: Not all cup configurations come with Adulteration Strip.

Adulteration Strip
See color chart and package insert for interpretation

5 Read Drug Strip at 5 minutes. Do not read the test results after 5 minutes.

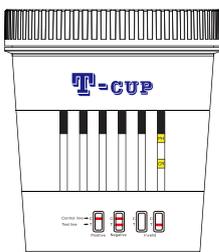


Read results at 5 minutes.

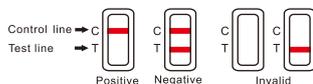

6 Technician dates and initials the security seal and places security seal over cap.

Date _____ Initial _____ Security Seal

Place security seal over cup

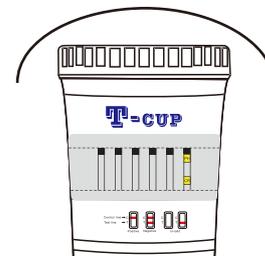

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)