


Introducing NUVETEST™ – Fast, accurate and convenient.

A one-step detection test for vaginitis associated with bacterial vaginosis (BV) and trichomonas.



NUVETEST™ effectively measures vaginal acidity associated with BV due to its buffer capacity

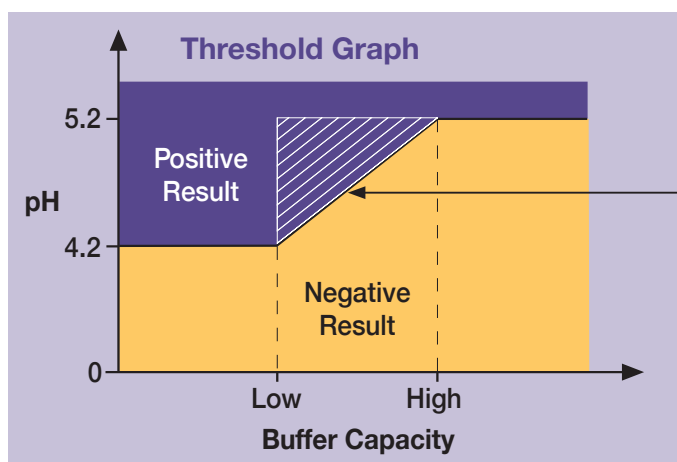


The inclusion of buffer capacity allows NUVETEST™ to capture those patients misdiagnosed with pH paper alone.

• Unlike regular pH tests which miss about 22% of BV cases with normal pH levels, NUVETEST™ improves detection accuracy to approximately 90% for added assurance.¹

Interpretation of NUVETEST™ results


Positive Result

- If the tip of the NUVETEST™ swab is stained or partially stained blue or green, vaginal acidity is disordered and infections associated with high pH are quite probable.
- Additional testing for bacterial vaginosis (BV) and trichomonas should be considered.

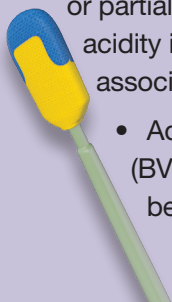

Negative Result

- If after **10 seconds** the NUVETEST™ tip is not stained, ie, completely yellow, vaginal acidity is normal.
- The risk of having an infection associated with elevated pH levels is unlikely.

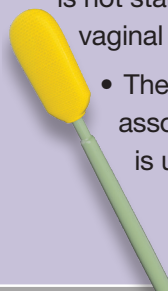

Important: If the patient is pregnant and the tip of the NUVETEST™ swab is stained blue or green, the color change may indicate an amniotic fluid leak.

Please see Warnings and Precautions on reverse side.

 NUVETEST™

Introducing NUVETEST™ – now available in full supply!



- Rapid results in 10 seconds
- Approximately 90% accuracy¹
- No need for color scale comparison, microscope work or delays related to lab tests



Rapid Results

Reimbursable
CPT Code: 83986

Clia Waived

Stock your shelves today with NUVETEST™ by using the ordering information below.

Product Description	Size	Ordering Number
NUVETEST™ Rapid Acidity Test for Evaluation of Vaginitis Item Number: 23710-008-25	25 Swabs	NUV-23710-008-25

Warnings and Precautions:

- Collect vaginal discharge from the anterior fornix or lateral vaginal wall of the mid vagina. Do not insert the NUVETEST™ further than 5 cm (2") in the vagina, to avoid contact with the cervix mucus, where pH levels are known to be higher than normal vaginal pH.
- Do not use lubricants which alter vaginal acidity.
- If the patient is pregnant, the stain may indicate an amniotic fluid leakage.

To ensure valid results, do not use NUVETEST™ in any of the following conditions:

- a) Less than 1 day before or the day after the patient's menstrual period;
- b) There are signs of menstruation or any vaginal bleeding;
- c) Less than 12 hours after sexual intercourse or use of vaginal products such as douching solutions or medications which alter vaginal acidity.

*Associated with Bacterial Vaginosis (BV), Trichomonas, and Vaginal Candidiasis

1. NUVETEST™ [prescribing information]. Exeltis USA, Inc.: Florham Park, NJ; 2018.

Please see full Information and Product Package Insert at **NUVETEST.COM**



NUVETEST™ is a trademark of Exeltis Dermatology, LLC.
© 2019 All rights reserved. Exeltis USA, Inc. EXP-19-0020 R00