

# Acucy® Influenza A&B Test

## HELPING YOU HIT THE MARK WITH INFLUENZA TESTING

The **Acucy® Influenza A&B Test** on the Acucy® System provides clinicians flexibility in workflow and accurate, standardized results for improved patient care.

## **Features and Benefits**

- Accurate Best-in-class influenza A test performance\*
- CLIA Waived Easy to use
- Standardized digital results on a reader
- Flexible Read Now and Walk Away modes
- Exceptional customer service Online training modules and technical support staffed by certified medical technologist/lab professionals

Performance Characteristics						
	Sensitivity	Specificity	Sample Type	Limit of Detection**	Time to result	
Туре А	96.4%	96.0%	Nasal/Nasopharyngeal	1.4 x 10 <sup>1</sup> TCID <sub>50</sub> /mL	15 minutes	
Туре В	82.3%	98.1%		2.4 x 10 <sup>1</sup> TCID <sub>50</sub> /mL		

<sup>\*</sup>When compared to immunoassay products in market

<sup>\*\*</sup>See IFU







## **SPECIFICITY**



# Acucy® Influenza A&B Test

## SIMPLE TO USE / RESULTS IN MINUTES




EXTRACT PATIENT SPECIMEN





SQUEEZE SIDES OF VIAL AND PULL OUT THE SWAB TO REMOVE EXCESS LIQUID

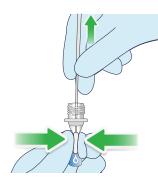



ADD DROPPER TIP



ADD DROPS INTO CASSETTE





#### **RUN TEST**

#### WALK AWAY/NORMAL MODE:

Reader will time test and display results

- · Remove cassette from foil pouch
- Place cassette in Reader
- Add sample to cassette
- Close reader drawer
- Reader will time test for 15 minutes
- Reader displays results

#### **READ NOW MODE:**

Test timed outside of the Reader

- Remove cassette from foil pouch
- Add sample to cassette
- Set timer for 15 minutes
- Place cassette into Reader
- Close Reader drawer
- Reader displays results





Ordering Information						
Description	Configuration	Catalog Number	CPT Code			
Acucy Influenza A&B Test Kit	25 Test (plus 2 for QC)	1010	Flu A 87804QW Flu B 87804QW-59			
Acucy Influenza A&B Control Kit***	5 Pos A/5 Pos B	1011	N/A			
Acucy System-Reader and Printer	1 each	1039	N/A			
USB Memory Drive****	1 each	1021	N/A			
Calibration Device****	1 each	1031	N/A			
Printer Roll****	1 each	1019	N/A			

<sup>\*\*\*</sup>Controls included with test kit but may be ordered separately

### THE AMERICAS

SEKISUI Diagnostics, LLC One Wall Street Burlington, MA 01803 Phone: 800 332 1042 Fax: 800 762 6311

Email: <u>questions@sekisui-dx.com</u>

### INTERNATIONAL

SEKISUI Diagnostics (UK) Limited Liphook Way, Allington Maidstone, Kent, ME16 OLQ, UK Phone: +44 1622 607800

Fax: +44 1622 607801 Email: info@sekisui-dx.com



Because every result matters™

sekisuidiagnostics.com

<sup>\*\*\*\*</sup>Included with Acucy System but may be ordered separately