



The world's leading rapid drug screening test cup.

Alere iCup[®] & iCup[®] A.D.

Urine Drug Screening Device



For professional in vitro diagnostic use only.


Better information. Better decisions.

Alere iCup® Urine Drug Screen

Features & Benefits

- Integrated test cup screens for 3-13 prescription and illicit drugs
- 510(k) cleared to market
- Fast results in 5 minutes
- Self-contained cup minimizes collector exposure to urine
- Specimen validity combinations available: CR, GL, NI, OX, pH, SG*

How it works

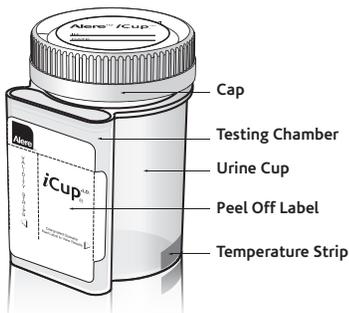

1
Collect specimen and secure the lid.



2
Peel label to reveal drug test and specimen validity (if applicable)



3
Interpret results at 5 minutes.



PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations, and warnings. For professional in vitro diagnostic use only. This assay provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography-mass spectrometry (GC-MS) or liquid chromatography-tandem mass spectrometry (LC-MS/MS) are the preferred confirmatory methods. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

Urine Screening Device Cutoff Levels

Test	Calibrator	Cutoff Level
Amphetamine (AMP)	d-Amphetamine	1,000 ng/mL
Barbiturates (BAR)	Secobarbital	300 ng/mL
Benzodiazepines (BZO)	Oxazepam	300 ng/mL
Buprenorphine (BUP)	Buprenorphine	10 ng/mL
Cocaine (COC)	Benzoylcegonine	300 ng/mL
Cocaine (COC 150)	Benzoylcegonine	150 ng/mL
Marijuana (THC)	11-nor- Δ^9 -THC-9-COOH	50 ng/mL
Methadone (MTD)	Methadone	300 ng/mL
Methamphetamine (mAMP)	d-Methamphetamine	1,000 ng/mL
Methamphetamine (mAMP 500)	d-Methamphetamine	500 ng/mL
Methylenedioxymethamphetamine (MDMA) Ecstasy	d,l-Methylenedioxymethamphetamine	500 ng/mL
Opiate (MOP)	Morphine	300 ng/mL
Opiate (OPI)	Morphine	2,000 ng/mL
Oxycodone (OXY)	Oxycodone	100 ng/mL
Phencyclidine (PCP)	Phencyclidine	25 ng/mL
Propoxyphene (PPX)	Propoxyphene	300 ng/mL
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000 ng/mL

*Specimen validity tests: Creatinine (CR), Glutaraldehyde (GL), Nitrate (NI), Oxidants (OX), pH (PH), Specific Gravity (SG)



Products 800.340.4029 | aleretoxicology.com

© 2014 Alere. All rights reserved. The Alere Logo, Alere, and iCup are trademarks of the Alere group of companies. 89001 REV4 10/14

