

# Power of **Fentanyl** Testing is in your hands!

*First handheld FIA reader for Fentanyl testing to be approved by the FDA*

510(k)  
cleared

Ultra Sensitive  
**1 ng/mL**



# VivaChek

BiōSieve™ / Toxismart™ / SalTox™



VivaChek Biotech (Hangzhou) Co., Ltd.  
Level 2, Block 2, 146 East Chaofeng Rd., Yuhang Economy  
Development Zone, Hangzhou, 311100, Zhejiang, P.R. China  
VivaChek Diagnostics, Inc. 1401 21st St. STER, Sacramento, CA  
TEL: (888) 860-3894  
E-mail: [info@vivachek.com](mailto:info@vivachek.com)  
Website: <https://vivachekdx.com>

Number: 1606017201  
Effective Date: 2024-06-25  
©2024 VivaChek Biotech (Hangzhou) Co., Ltd.

# VivaChek


01

# FIA TESTING SYSTEM


## Fentanyl Test Procedure




1 Press the right button “▶” for 5 seconds to turn on the Reader.

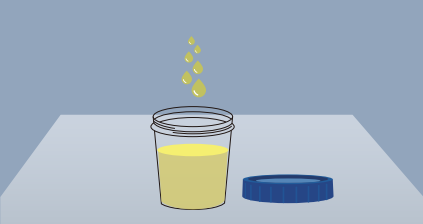

2 Collect the urine specimen.

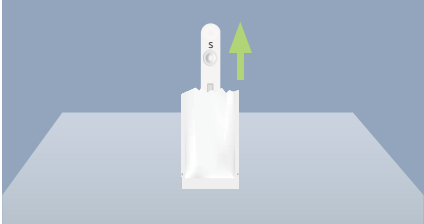


3 Take the Test Device out of the foil pouch and place it on a clean, dust-free and flat surface.

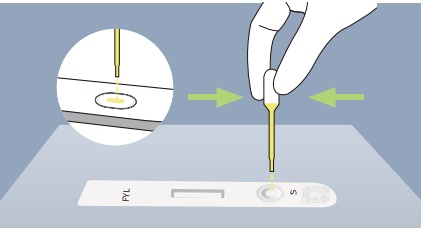

4 Transfer 3 drops of urine sample vertically to the sample well (S) using the dropper.



5 Insert the Test Device into the slot of the Reader immediately Please ensure that orientation of the Test Device is as shown in the image above. .

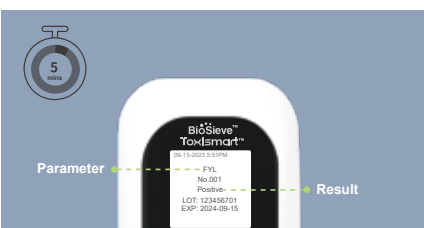

6 The Reader will automatically display the result after 5 minutes countdown.

### Product information

|               |                       |
|---------------|-----------------------|
| Product       | Fentanyl FIA Test Kit |
| Specimen      | Urine                 |
| Reaction time | 5 min                 |
| Cut-off level | 1.0 ng/mL             |

|              |                      |
|--------------|----------------------|
| Product      | Toxismart FIA Reader |
| Size         | 124.5 × 72.5 × 40 mm |
| Weight       | 165 g                |
| Connectivity | Bluetooth            |
| Memory       | 999 results          |



### Ordering information

| Item code (REF) | Description                     | Size             |
|-----------------|---------------------------------|------------------|
| VCUS-FIARDR     | BioSieve™ Toxismart FIA Reader  | 1Pcs/Box         |
| VC-FYL1         | BioSieve™ Fentanyl FIA Test Kit | 25T/Kit          |
| VC-FYCTL        | BioSieve™ Fentanyl Controls     | 1 Neg &1 Pos/Box |