



## CLIAwaived,™ Inc. INSTANT DRUG TEST CARD II

Test up to 14 drugs and 4 adulterants on the same test card. The test card allows for an easy mix and match of tests.

The newly designed IDTC-II dip card features hermetic seal to prevent card from flooding. The device features flat design and separate cap so card can be easily photographed without contamination.



- ✓ **CLIA waived for 14 drugs**
- ✓ **Rapid results, read between 5-60 minutes**
- ✓ **Room temperature storage for up-to 18 months**

PANELS	ADULT	AMP	BAR	BUP	BZO	COC	EDDP	MDMA	MET	MTD	OPI	OXY	PCP	TCA	THC	ITEM #
5				■	■			■		■					■	CLIA-IDTC-II-05-3571
6	■			■	■			■		■					■	CLIA-IDTC-II-06-264
9				■	■	■		■	■	■	■	■			■	CLIA-IDTC-II-09-23
		■	■	■	■	■		■	■	■					■	CLIA-IDTC-II-09-14
10		■	■	■	■			■	■	■	■				■	CLIA-IDTC-II-10-55
12		■	■	■	■			■	■	■	■	■			■	CLIA-IDTC-II-88
	■	■	■	■	■			■	■	■	■	■			■	CLIA-IDTC-II-88A
14		■	■	■	■			■	■	■	■	■	■	■	■	CLIA-IDTC-II-14-01

# INSTRUCTIONS FOR OTC USE:

## BEFORE TESTING

Read the instructions completely.

Check the expiration date on the box. Do not use the test if it is expired. Have a watch, clock or timer ready.

The following items are needed only if you choose to ship samples for confidential confirmation lab testing:

- Pre-addressed shipping box
- Plastic transportation bag
- Identification label

## PERFORMING THE TEST

### Step 1: Take Out the Test Device

Take the test device from the sealed foil pouch. The CLIAwaived, Inc. Instant Drug Test comes in two (2) types: dip card and cup.

### Step 2: Apply Urine to the Test Device

#### **If Using a Dip Card:**

1. Remove the dip card from the sealed pouch. Write the donor name or ID in the provided space and then remove the cap.
2. With the arrows pointing toward the urine specimen, **immerse the sample tips vertically** in the urine specimen for **at least 20 seconds**. Put the cap back on the dip card. Place the dip card on a flat surface.



#### **If Using a Cup:**

1. Remove the cup from the sealed pouch. Write the donor name or ID in the space provided.
2. **Collect urine** in the cup.




### **Step 3: Read Result**

Read results after 5 minutes. Do not wait longer than 60 minutes.

A red or pink line must appear next to the letter “C” (control) on all of the test strips. The appearance of a red or pink next to the letter “C” on each test strip indicates that the test has worked properly. If you see control lines on all the test strips, you can read your test results.

#### **Negative Result:**


A red or pink line next to the “T1” or “T2” (drug test line) under the drug name indicates a negative result for that drug. If a test line appears next to the “T1” or “T2” for all drugs, the sample is considered negative. Certain lines may appear lighter or thinner than other lines.

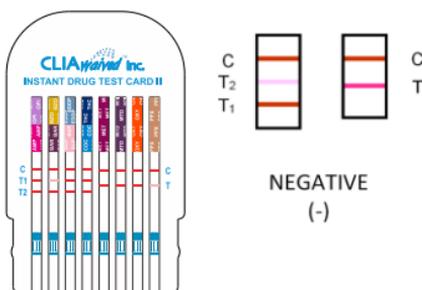

#### **Preliminary Positive Result:**

If NO red or pink line appears next to the “T1” or “T2” under the drug name, the sample may contain that drug. Send the sample to a laboratory for confirmation testing.

*The illustration on the right shows preliminary positive results for OPI, COC and OXY, but negative for all other drugs.*

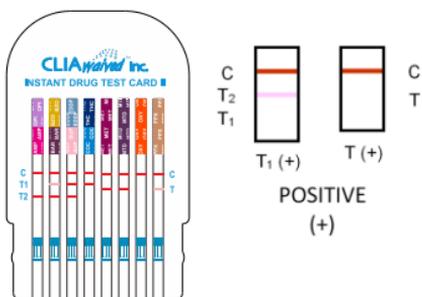

#### **Invalid Result:**

A colored line should always appear next to the letter “C” on every test strip. If no control line appears on any of test strips, the result is invalid.

*The illustration at right shows no line next to the letter “C” on the first strip (OPI, AMP), third strip (EDDP, BUP) and seventh strip (OXY). The test results for those three test strips are invalid.*

