




Complete Drug Test Cup "CDTC" Procedure Card

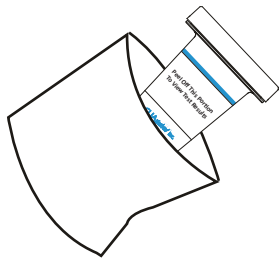


1. Remove the cup from the foil pouch.

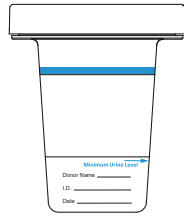

2. Write donor ID and date on the label.

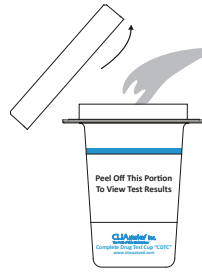

3. Collect specimen in the cup.

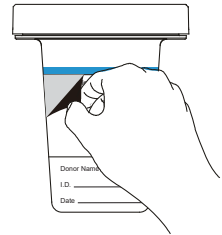

4. Peel off the label and read results. Read the drug test result at 5 minutes. Do not read after 10 minutes.

INTERPRETATION OF RESULT

Each strip contains one drug test. C region shows validity of a test result. T region shows result for this test.

For C region:

The appearance of a line indicates a valid result.


No line means an Invalid result.

If a test strip does not have a line in the C region, test results are Invalid for this strip.

For T region:


The appearance of a line indicates a Negative result.

Note: Any test line, even a very faint test line, is considered a negative result.

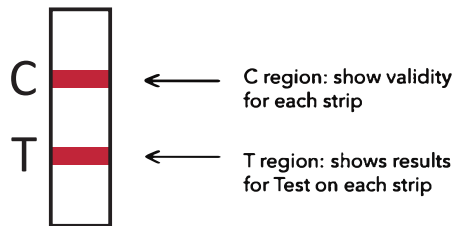

No line indicates a Preliminary Positive result.

Note: Any urine with preliminary positive results should be sent to a laboratory for confirmation.

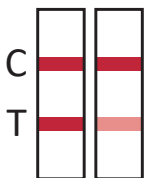


(Negative Result)

A red or pink line next to the "T"(drug test line) under the drug name indicates a negative result for that drug. Certain lines may appear lighter or thinner than other lines.



(Preliminary Positive Result)

If NO red or pink line appears next to the "T" under the drug name, the sample may contain that drug.

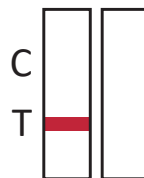

(Invalid Result)

A colored line should always appear next to the letter "C" on every test strip. If no control line appears on any of test strips, the result is invalid.

	No adulteration	Adulteration
AD1 (OX)		
AD2 (S.G.)		
AD3 (pH)		
AD4 (NIT)		
AD5 (GLU)		
AD6 (CRE)		

Read adulteration test results at 3 - 5 min. 143028902

Semi Quantitative results are obtained by visually comparing the reacted color blocks on the strip to the printed color blocks on the color chart.

CLIAwaived,™ Inc. Complete Drug Test Cup "CDTC"



DETECTS UP TO 18 DRUGS

Amphetamine (AMP)
Buprenorphine (BUP)
Secobarbital (BAR)
Benzodiazepine (BZO)
Cocaine (COC)
Methamphetamine (MET)
Methylenedioxyamphetamine (MDMA)
Morphine (MOP/OPI)
Methadone (MTD)
Oxycodone (OXY)
Phencyclidine (PCP)
Nortriptyline (TCA)
Marijuana (THC)
Fentanyl (FYL)
2-ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)
Tramadol (TRA)
Propoxyphene (PPX)
6-monoacetylmorphine (6-MAM)

+ adulteration
(OX/SG/pH/NIT/GLU/CRE)

- Fast, dark test lines
- FDA Cleared and CLIA-waived
- Results in 5 minutes
- Built-in temperature strip and 3 test strips for adulterants
- Remarkable quality and pricing