



Statement

To Whom It May Concern,

We, Hangzhou AllTest Biotech Co., Ltd. Hereby certify that:

DUA-A1147C:Multi-Drug14 Drugs Rapid Test Cup-CLIA Waived (Urine)
AMP500/BAR300/BZO300/BUP10/COC150/THC50/MTD300/MET500/MDMA500/M
OP300/PCP25/TCA1000/OXY100/FYL1+(OXI/CRE/pH),25T/carton.

A false positive result for Methadone (cut-off: 300 ng/mL) may occur when Disopyramide is present at concentrations between 2000-3000 ng/mL. It should be noted that Disopyramide was not evaluated during development; consequently, this cross-reactivity was not discovered and, as such, has not been included in the IFU.

R&D Manager: Darren Wang

Signature:

Date: November 21st 2025

Hangzhou Alltest Biotech CO., Ltd.

Add:#550 Yin Hai Street, Hangzhou Economic and Technologic Development Area,
310018, P.R China