

*Name: *Organization:
*Address:
*City: *State: *Zip:
*E-mail:
*Phone: Fax:

**Denotes required fields. Your test will not be evaluated until these fields are populated.*

Instructions: This is a multiple-choice test. Read the questions completely before choosing the best answer. Click on the button that corresponds to the best answer. You must complete the test with a score of 80 percent or better to become certified.

1. The CLIAwaived, Inc. Instant Drug Test Cup (IDTC II) provides

- ☐ a. Preliminary drug test results
- ☐ b. quantitative drug test results
- ☐ c. qualitative drug test results
- ☐ d. confirmed test results
- ☐ e. both a and c are correct

2. The presence of a colored line in the control region and a colored line in both test regions (T1 and T2) indicates that

- ☐ a. the test is invalid
- ☐ b. the test is negative
- ☐ c. the test is presumptive positive

3. The absence of a colored line in the control region (C) indicate

- ☐ a. the test is invalid
- ☐ b. the test result is negative
- ☐ c. the test result is presumptive positive
- ☐ d. the test should be repeated using a new device
- ☐ e. both a and d are correct

4. The CLIAwaived, Inc. Instant Drug Test Cup (IDTC II) should be stored

- ☐ a. in the freezer
- ☐ b. at 2-30 C (36-86 F) (either room temperature or the refrigerator)
- ☐ c. only in the refrigerator

5. Test results can be read up to:

- ☐ a. 2 days
- ☐ b. 1 hour
- ☐ c. 15 minutes
- ☐ d. Between 5-60 minutes

6. Provided the control line appears, it is ok to use test cups past their stated expiration date

- ☐ a. Yes
- ☐ b. No

(IDTC II) Training Quiz Continued

7. Test results can be read up to:

The color intensity of a line at the "T" region must be as dark as the color intensity of the line at the "C" region.

- ☐ a. True
- ☐ b. False

8. You should wait 5 minutes before reading the test results

- ☐ Yes
- ☐ No

9. A "ghost" line with valid control lines should be interpreted as

- ☐ a. Presumptive positive
- ☐ b. Negative

10. External quality controls are recommended to ensure proper kit performance

- ☐ a. True
- ☐ b. False

PRINT AND FAX THIS TEST TO **801.720.7568**