

# SURE CHECK® HIV 1/2 Assay

A unique, easy-to-use, self-contained collection and testing device for the rapid, visual detection of antibodies to HIV-1 and HIV-2.

## Simple

- CLIA waived<sup>1</sup>
- Sample collection and test device in one
- 2-step test procedure
- Room temperature storage:  
46° to 86°F (8° to 30°C)



## Reliable

- FDA approved
- Sensitivity: 99.7%; Specificity: 99.9%<sup>2</sup>
- Built-in IgG procedural control

## Fast

- Rapid results: 15 minutes

**Smallest sample size: 2.5 µL**



Distributed by [CLIAwaived.com](http://CLIAwaived.com)  
[Orders@cliawaived.com](mailto:Orders@cliawaived.com)  
Call us at: 1-888-882-7739

 **CHEMBIO**  
DIAGNOSTIC SYSTEMS, INC.  
*Rapid Tests for Earlier Treatment™*

<sup>1</sup> CLIA waived for fingerstick and venous whole blood


<sup>2</sup> See product insert for complete performance details

**Manufactured in the USA**

# 2-Step Procedure\*



## 1 ADD SAMPLE

Prick finger using lancet. TOUCH blood drop with capillary tip of sampler.

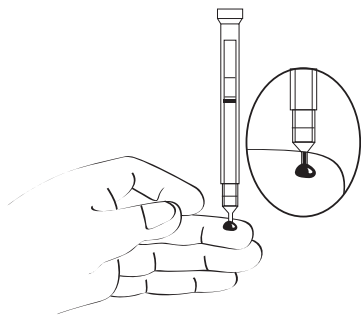


## 2 RUN TEST

PUSH capillary tip of sampler completely into buffer vial. Read the test between 15 and 20 minutes.

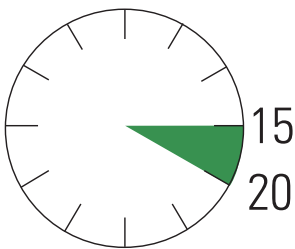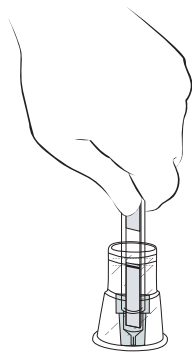

## Possible Results

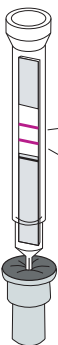

Control Line  
HIV 1/2 Reactive

NON-REACTIVE




1 line

REACTIVE



2 lines

INVALID

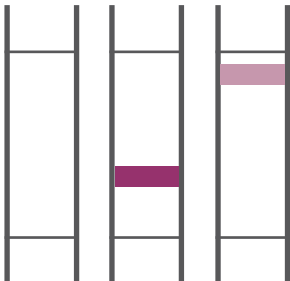

No control line, or  
outside area

INVALID



Test line  
outside area

\* Before testing, please refer to the complete product insert provided in the test kit

## Ordering Information

Reimbursement Code: 86703-QW, G0433-QW

Description	Contents	Catalog Number
SURE CHECK® HIV 1/2 Assay FDA Approved, CLIA Waived/Non-Waived	25 Individually Pouched Test Devices (Including a Sterile Lancet, Bandage, and Desiccant Packet) 25 Disposable Test Stands 25 Copies of Subject Information Notice 1 Quick Reference Instructions (for CLIA Waived) 1 Product Insert	60-9507-0
HIV Rapid Test Control Pack FDA Approved	One Pack Contains: - 1 HIV-1 Reactive Control 0.25 mL - 1 HIV-2 Reactive Control 0.25 mL - 1 HIV Nonreactive Control 0.25 mL Each kit control vial contains sufficient volume to run up to 100 tests	60-9549-0

SURE CHECK® HIV 1/2 Assay is sold in the EU as STAT-VIEW® HIV 1/2 Assay

Call us at: 1-888-882-7739

Orders@cliawaived.com

Distributed by CLIAwaived.com

MS-16-001