

CLIAwaived,™ Inc. RAPID STREP A TEST

Group A Strep is one of the most serious cases of upper respiratory infections and the major cause of Pharyngitis, which, if left untreated, can lead to serious complications such as rheumatic fever, acute sinusitis, scarlet fever and glomerulonephritis. The CLIAwaived,™ Inc. Rapid Strep A Test is a lateral flow, membrane-based immunoassay for the qualitative detection of Group A Streptococcal antigen from throat swab specimens.

- Results in 5 minutes
- No cross reactivity with saliva
- Room temperature storage with 18-month shelf life



CLIA-ST-501 (30 tests)

Note: This test is for waived labs with a current Certificate of Waiver.

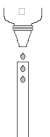
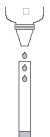
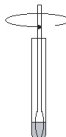
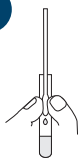


CPT CODE: 87880QW

National limit Amount: \$16.88



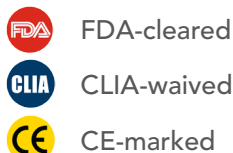
RAPID STREP A TEST

The **CLIAwaived,™ Inc. Rapid Strep A Test** is a rapid, visual test for the presumptive *in-vitro* qualitative detection of Group A Streptococcal antigen from throat swab specimens. This test is for professional use only.

- 1  Add 3 drops of Reagent A to the extraction tube.
- 2  Add 3 drops of Reagent B to the extraction tube.
- 3  Add specimen swab into extraction tube. Mix solution thoroughly by swirling swab vigorously in extraction tube several times. Allow swab to soak for at least 1 minute, but no longer than 15 minutes.
- 4  Remove all liquid from swab by rolling it against the side of the extraction tube and squeezing the swab between the flexible sides. Discard swab in a biohazard container.
- 5  Place the dipstick into the extraction tube. Gently stir for 10-15 seconds.
- 6  Allow the dipstick to remain in tube to absorb as much of extraction liquid as possible. Read results after 5 minutes. Do not interpret results after 10 minutes.

 Please refer to package inserts for detailed instructions and technical information.

DISCLAIMER – The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for health care services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived™ Inc. All CPT codes are supplied for information purposes only and represent no statement, promise or guarantee by CLIAwaived™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Associations. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercial s. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.



CLIAwaived™ Inc.

For more info, please call: **888.882.7739**

Visit us online at: **www.cliawaived.com**