

# osom<sup>®</sup>

## Trichomonas Rapid Test



THE ONLY CLIA-WAIVED RAPID TEST FOR THE DETECTION OF *TRICHOMONAS VAGINALIS*


FOR USE WITH VAGINAL SWABS

### FEATURES:

- ✓ **ACCURATE;** 83% sensitivity vs. culture; 95% agreement with culture and wet mount combined
- ✓ **FAST;** Results in 10 minutes or less
- ✓ **OBJECTIVE;** Detects the antigen; removes uncertainty associated with live organism instability
- ✓ **WORLD-CLASS TECHNICAL SUPPORT;** Staffed with certified medical technologists and skilled professionals




### TEST PROCEDURE / RESULTS

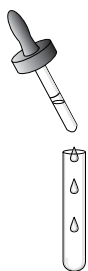

Add sample buffer to test tube.

Insert swab and mix.

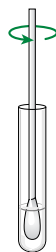


1 MIN



Let stand 1 minute.

Remove swab from test tube.

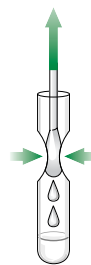

Insert test stick into extracted sample.



10 MIN



Read results at 10 minutes.

**POSITIVE**



**NEGATIVE**



### ORDERING INFORMATION

- ☐ OSOM<sup>®</sup> TRICHOMONAS RAPID TEST  
TRICHOMONAS POSITIVE CONTROL KIT

### CONFIGURATION

25 Tests

### CATALOG NUMBER

181 OR 181E (CE marked)  
182

**genzyme**  
Diagnostics

Distributed by: **CLIAwaived.com<sup>™</sup>**  
San Diego, CA 92121  
tel 858-481-5031  
toll free 888-882-7739  
www.cliawaived.com

