


# SAS™ Adeno Test

CLIA  
NON-WAIVED

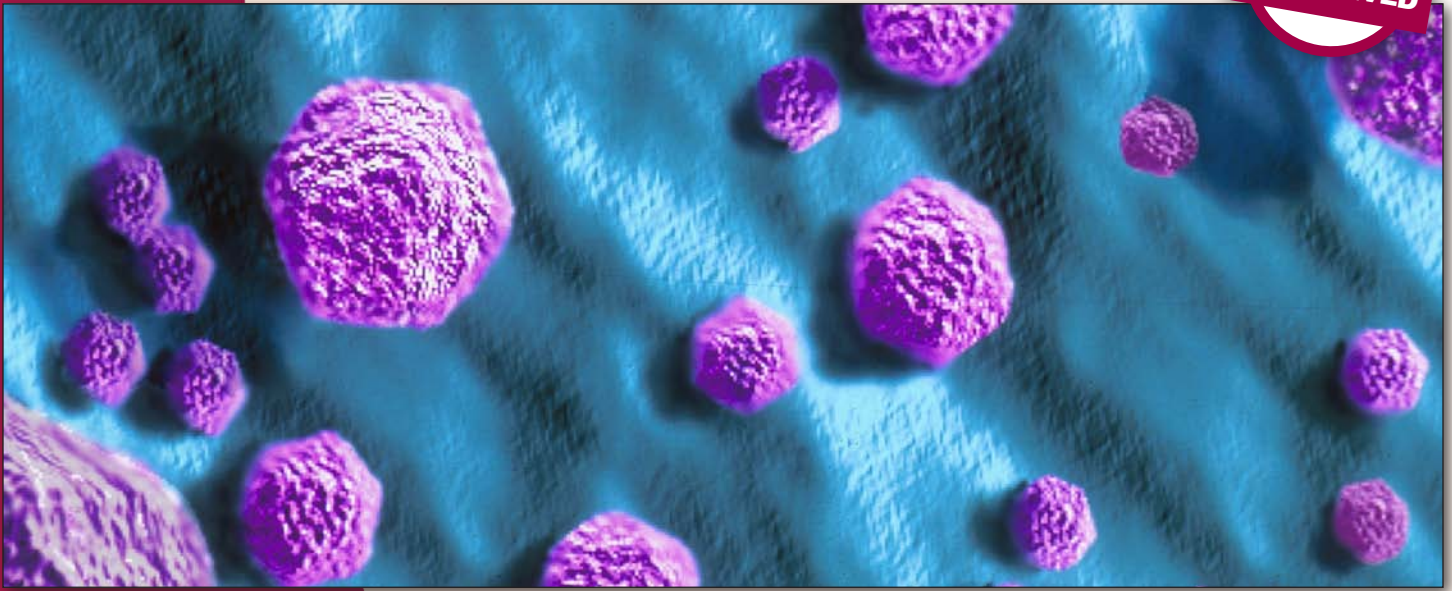

SAS™ Adeno Test is a rapid, membrane-based immunoassay for the qualitative detection of Adenovirus and Adenovirus antigen. Test aids in the diagnosis of conjunctivitis seen in ocular infections, and respiratory illnesses, and gastroenteritis caused by Adenovirus.

- ▶ **CPT Code:**  
87809
- ▶ **Sensitivity:**  
96.8%
- ▶ **Specificity:**  
94.1%
- ▶ **Specimen:**  
Cell Culture,  
Eye Swab, Nasal  
Pharyngeal  
Secretions,  
Fecal Material


- Clia Classification: Non-waived
- Based on the SAS™ Rapid Immunochromatographic Assay utilizing a combination of monoclonal antibody reagents to detect Adenoviruses.
- Simple to run--results with 15 minutes
- Buffer and extraction tubes are provided in kit
- Room temperature storage
- 18-month shelf life



# SAS™ Adeno Test

## Test Procedure

Acceptable specimens include cell culture, eye swab, nasopharyngeal secretions, and fecal matter. Allow specimen and/or controls to reach room temperature prior to testing.

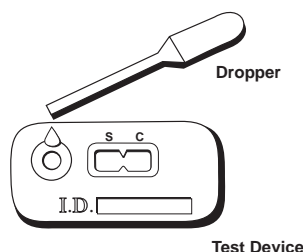

1. Remove the test device from the protective pouch and place it on a flat surface. Label the device with patient or control identifications.
2. Dispense 3-4 drops (approximately 150µL) of the specimen into the round sample well (see illustration). Wait for colored lines to appear.



NOTE: Swabs or samples must be extracted using the provided extraction buffer.

## Test Results



Read results after 15 minutes and no later than 30 minutes. Positive results may be observed in as short as 30 seconds depending on the concentration of the antigen.

The test is negative if a colored line only appears in the C (control) area. The test is positive if one colored line appears in the C (control) area and one line appears in the S (specimen) area.

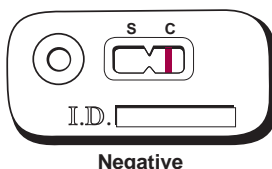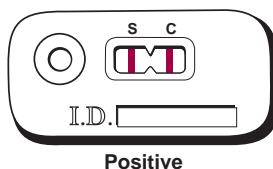

(See package insert for detailed instructions.)

Product No.

**067020**

20-Test Kit

Product No.

**067101**

Control

(2 × 1 mL,  
2 Vials/Pk)

Description	Cat. No.	Test/ Kit	Specimen	Application/Shelf Life	Sensitivity, Specificity	CLIA/ CPT Code	Results/ Minutes
SAS™ Adeno Test	067020	20	Cell Culture/Eye Swab/ Nasopharyngeal Secretions/ Fecal Material	Detects all Adenovirus strains (Shelf Life: 18 mos.)	96.8%, 94.1%	Non-waived/ 87809	15
Control Set	067101	2 × 1 mL, 2 Vials/Pk (NOTE: For use with 067020)					

Distributed by:

**CLIAwaived™**.com

For more information please contact CLIAwaived Inc. at  
11578 Sorrento Valley Road ■ San Diego, CA 92121  
Tel: (858) 481-5031 ■ Toll Free: (888) 882-7739  
E-mail: [info@cliawaived.com](mailto:info@cliawaived.com)  
Visit us on the web at: [www.cliawaived.com](http://www.cliawaived.com)  
Distributed by: CLIAwaived, Inc.

Manufactured by:



**SA Scientific™**

Reliable Diagnostic Tests

4919 Golden Quail, San Antonio, Texas 78240 U.S.A.  
(210) 699-8800 • (210) 699-6545 Fax • [www.sascientific.com](http://www.sascientific.com)