

**NOW  
DETECTS  
BUP!**

## CLIAwaived,™ Inc. SINGLE DIP DRUG TEST

When quality and cost is your target, this test is the answer. This multiple-drug dipstick is designed for its simplicity of use and reasonable cost. It's user-friendly, reliable and accurate.

- One step qualitative assay
- Results in five minutes
- Room temperature storage



### SINGLE DIP DRUG TEST

### PACKAGING: 50 TESTS

1-PANEL SDDT			4-PANEL SDDT	
Tests for:	Sensitivity (ng/mL)	Catalog #	Tests for:	Catalog #
Marijuana (THC)	50	CLIA-SDDT-10	Barbiturates (BAR)	CLIA-SDDT-QF08
Cocaine (COC)	300	CLIA-SDDT-11	Benzodiazepines (BZD)	
Opiates (OPI)	300	CLIA-SDDT-13	Methadone (MAD)	
Amphetamine (AMP)	1000	CLIA-SDDT-14	Oxycodone (OXY)	
Methamphetamine (MET)	500	CLIA-SDDT-16	Marijuana (THC)	CLIA-SDDT-QF09
Phencyclidine (PCP)	25	CLIA-SDDT-17	Cocaine (COC)	
Benzodiazepines (BZD)	300*	CLIA-SDDT-18	Opiates (OPI)	
Barbiturates (BAR)	300*	CLIA-SDDT-19	Benzodiazepines (BZD)	
Methadone (MAD)	500	CLIA-SDDT-20	Marijuana (THC)	CLIA-SDDT-QF10
Tricyclic Antidepressants (TCA)	1000*	CLIA-SDDT-21	Cocaine (COC)	
Ecstasy (MDMA)	500	CLIA-SDDT-22	Opiates (OPI)	
Oxycodone (OXY)	100*	CLIA-SDDT-23	Methadone (MAD)	
Buprenorphine (BUP)	10	CLIA-SDDT-25	Marijuana (THC)	CLIA-SDDT-QF12
			Cocaine (COC)	
			Opiates (OPI)	
			Methamphetamine (MET)	


**Reimbursement Information: CPT Code # G0434QW, 80101, 80104– Drug screen, other than chromatographic; any number of drug classes, by CLIA-waived test or moderate complexity test, per patient encounter. National Limit Amount: \$20.47.**

DISCLAIMER: The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for healthcare services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived,™ Inc. All CPT codes are supplied for information purposes only and represent no statement, promise or guarantee by CLIAwaived,™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Associations. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercial payors. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

\*NOTE: There are no uniformly recognized cutoff levels for drugs of abuse testing for Benzodiazepine, Oxycodone, Tricyclic Antidepressant and Barbiturates. The test is not intended to screen or monitor individuals who are prescribed these drugs by a physician; the test may yield positive results for individuals taking such drugs, as prescribed.

# SINGLE-DIP DRUG TEST


The **CLIAwaived,™ Inc. Single Dip Drug Test\*** provides visual qualitative results and is intended for *in-vitro* diagnostic use in both laboratory and non-laboratory CLIA-waived settings. It is not intended for over-the-counter sale to non-professionals.

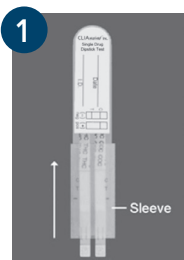

1 Open pouch and remove device.  
Write the urine sample ID number on the device.  
Push the sleeve all the way up.




2 Dip the device into the urine sample for 10 seconds.  
Do not dip above the tip of the arrows.

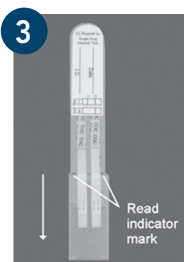

3 Push sleeve down to the Read Indicator Mark.  
Place the device on a flat surface.  
Interpret results at 5-30 minutes.\*  
Some devices have two sides. Be sure to check both sides for results.

 Please refer to package inserts for detailed instructions and technical information.

\*NOTE: The CLIAwaived,™ Inc. Single Dip Drug Test (SDDT) provides only preliminary test results for drugs-of-abuse. For a quantitative result or to confirm positive results obtained by the CLIAwaived,™ Inc. Single Dip Drug Test, a more specific alternative chemical method must be used. Gas Chromatography/Mass Spectrometry (GC/MS) is the preferred confirmatory method.

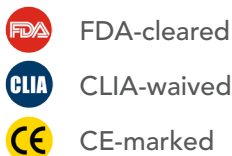

**CLIAwaived™ Inc.**

For more info, please call: **888.882.7739**

Visit us online at: **www.cliawaived.com**