

Fastect[®] II Certification Test

Instructions: This is a multiple-choice test. Read the questions completely before choosing the best answer. Click on the radio button that corresponds to the best answer. You must complete the test with a score of 80% or better to become certified.

* Note that Name, Organization, Address and Email are required fields. Your test will not be evaluated unless these fields are populated and depending on your browser settings, your answers may be lost if you try to submit without filling out the required fields.

*Name:

*Organization:

Address

*No.& Street

*City:

*State:

*Zip:

*Email:

*Phone:

Fax:

1. Fastect[®] II provides

- a. preliminary drug test results
- b. quantitative drug test results
- c. qualitative drug test results
- d. confirmed test results
- e. both a and c are correct

2. The absence of a colored band in the control region (C) means


- a. the test result is negative
- b. the test result is presumptive positive
- c. the test result is invalid
- d. the test should be repeated using a new device
- e. both c and d are correct

3. The presence of a colored band in the control region (C) and a colored band at a specific test means


- a. the test is invalid
- b. the test is negative for that particular drug
- c. the test is presumptive positive for that particular drug

4. The Fastect[®] II device should be stored at
- a. 25-35°C.
 - b. 35-77°F.
 - c. 15-30°C.
 - d. 59-86°F.
 - e. both c and d.

5. Interpret the following test
- a. Negative for THC
 - b. Presumptive positive for THC
 - c. Invalid

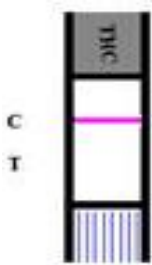

6. Interpret the following test
- a. Invalid
 - b. Negative for THC
 - c. Presumptive positive for THC

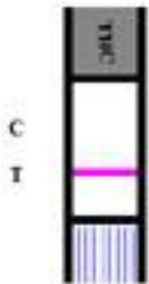

7. The device should be dipped into the urine sample for at least
- a. 10 seconds
 - b. 1 minute
 - c. 30 seconds
 - d. 3 minutes

8. Once the control band forms in _____ or less, the results are ready to be read.
- a. 5 minutes
 - b. 4 hours
 - c. 8 days
 - d. 1 hour
9. Fastect[®] II should be dipped into the urine up to the level
- a. indicated by the tips of the arrows
 - b. marked "T"
 - c. marked "C"
10. Which of the following are true?
- a. The presence of a band at the test region (T) indicates a presumptive positive result.
 - b. Do not compare band intensity of one test to another.
 - c. No band at the test region (T) indicates a negative result.
 - d. The presence of a colored band at the control region (C) and a colored band at a specific test (T) region indicate that the results are negative for that particular drug.
 - e. Both b and d.