

QuickTox® MULTIPLE DRUG DIPCARD

The QuickTox® Multiple Drug Dipcard is a multiple drugs-of-abuse dip card that allows for easy mix and match of tests. It is capable of testing up to 12 drugs (or nine drugs and four adulterants) simultaneously. With its flat design and clear cap, the device can be photographed without fear of contamination of the tester's desk.

- Minimized sample handling with results in minutes
- Built-in adulterant testing available
- Room temperature storage for 12-18 months



RAPID DIP DRUG TEST **													PACKAGING: 25 TESTS	
Tests for:	THC 50	COC 300	OPI 300	AMP 1000	MET 500	PCP 25	BZO 300*	BAR 300*	MTD 300	OXY 100*	MDMA 500	BUP 10	ADULTERANTS (CR • NI • PH • OX)	Catalog #
5 Panel	●	●	●		●		●							BRN-QT-13
	●	●	●	●	●									BRN-QT-19
8 Panel	●	●	●	●	●		●	●		●				BRN-QT-44
9 Panel	●	●	●	●	●		●	●	●	●				BRN-QT-52
	●	●	●	●	●		●	●	●	●			●	BRN-QT-52A
11 Panel	●	●	●	●	●	●	●	●	●	●	●			BRN-QT-70
12 Panel	●	●	●	●	●	●	●	●	●	●	●	●		BRN-QT-88

Reimbursement Information: CPT Code # G0434QW, 80104 – Drug screen, other than chromatographic; any number of drug classes, by CLIA-waived test or moderate complexity test, per patient encounter. National Limit Amount – \$20.47

*There are no uniformly recognized cutoff levels for drugs of abuse testing for Benzodiazepine, Oxycodone, Tricyclic Antidepressant and Barbiturates. The test is not intended to screen or monitor individuals who are prescribed these drugs by a physician; the test may yield positive results for individuals taking such drugs, as prescribed.

**NOTE: The QuickTox® Multiple Drug Dipcard provides only preliminary test results for drugs-of-abuse. For a quantitative result or to confirm positive results obtained by the QuickTox® Multiple Drug Dipcard, a more specific alternative chemical method must be used. Gas Chromatography/Mass Spectrometry (GC/MS) is the preferred confirmatory method.

DISCLAIMER – The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for healthcare services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived,™ Inc. All CPT codes are supplied for information purposes only and represent no statement, promise or guarantee by CLIAwaived,™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Association. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercial payors. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.



QuickTox® MULTIPLE DRUG DIPCARD

PROCEDURE INSTRUCTIONS

The **QuickTox® Multiple Drug Dipcard** is a one-step dip card with or without adulteration tests for the rapid qualitative detection of drugs of abuse.




Remove the test from the pouch by tearing at the notch. Remove the bottom cover to expose the test strips.



Dip the sample pads of the test device straight into the urine sample for a minimum of 10 seconds. Dip up to, but not beyond, the tips of the arrows.



Remove the test from the urine sample and reattach the bottom cover. Place the device on a level surface.

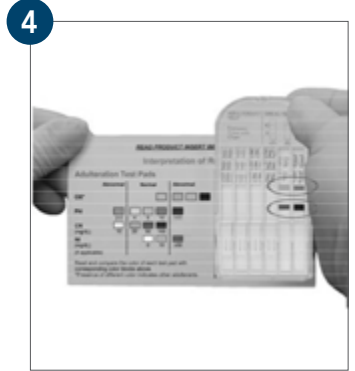

Adulteration Test: After 1 minute, compare colors of adulteration tests with corresponding color blocks on the color chart included in kit. Do not read after 2 minutes as color may change.



Drugs of Abuse Test: Read results once the control band (C) appears and the membrane clears (approx. 5 minutes). Interpret according to these instructions.

NEGATIVE RESULT

For each test, a colored band should be observed in the CONTROL (C) region and one in the TEST region. Any band that can be seen visually is a negative result.



Negative

POSITIVE RESULT

A colored band in the CONTROL (C) region should be observed. When no colored band is at the specific TEST region, the test is presumptive positive for that particular drug.



Positive for COC, THC

INVALID RESULT

When no band appears in the CONTROL (C) region, the test is invalid, even if there is a band in the test region. Check test procedures and samples, and repeat test with new device.




Invalid for OPI, AMP

⚠ Please refer to package inserts for detailed instructions and technical information.

Distributed by:

CLIAwaived™ Inc.

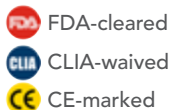

For more info, please call: **888.882.7739**.
Visit us online at: **www.cliawaived.com**.