

QuickTox[®] Certification Test

Instructions: This is a multiple-choice test. Read the questions completely before choosing the best answer. Click on the radio button that corresponds to the best answer. You must complete the test with a score of 80% or better to become certified.

* Note that Name, Organization, Address and Email are required fields. Your test will not be evaluated unless these fields are populated and depending on your browser settings, your answers may be lost if you try to submit without filling out the required fields.

*Name:

*Organization:

Address:

*No.& Street:

*City:

*State :

*Zip:

*Email:

*Phone:

Fax:

1. The QuickTox[®] provides for the detection of
 - a. 1 drug in oral fluid.
 - b. 6 drugs in urine and oral fluid.
 - c. Up to 13 drugs or 12 drugs and 6 adulterant tests in urine.

2. QuickTox[®] provides _____ drug test results.
 - a. Preliminary
 - b. Quantitative
 - c. Qualitative
 - d. Confirmed
 - e. Both a and c are correct


3. The absence of a colored band in the control region (C) means the test result is
 - a. Negative
 - b. Presumptive positive
 - c. Invalid

4. The presence of a colored band at the control region (C) and a colored band at a specific test means the test is

- a. Invalid
- b. Negative
- c. Presumptive positive


5. Interpret the following test:

- a. Negative for COC, THC, and MET
- b. Presumptive positive for COC, OPI, THC, AMP, PCP and BZO
- c. Invalid
- d. Both A & B

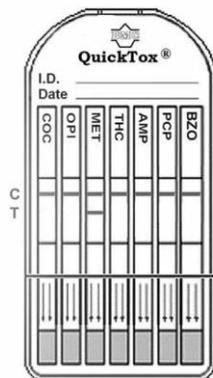

6. Interpret the following test

- a. Invalid
- b. Presumptive positive for AMP and OPI
- c. Negative for Drugs of Abuse

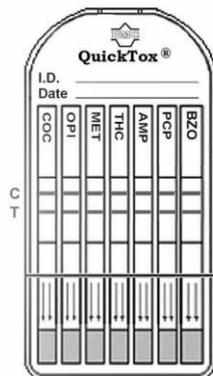

7. The QuickTox[®] device should be dipped into the urine sample for a minimum of
- 10 seconds
 - 1 minute
 - 30 days
 - 3 hours
8. Once the control band forms in _____ or less, the results are ready to be read
- 2 days
 - 1 hour
 - 8 hours
 - 5 minutes
9. The arrows indicate the level at which the strips must be dipped in the urine. Dip the QuickTox[®] device
- Until the urine covers the arrows entirely.
 - To the tip of the arrows.
 - To the edge of the clear plastic case.
10. Which is false pertaining to adulteration tests?
- They ensure the integrity of the urine samples in DAU testing.
 - Creatinine, Nitrite, Oxidizing Agents, pH, Specific Gravity and Glutaraldehyde are adulterant pads included in the QuickTox[®] Drug Screen Dipcard.
 - Adulteration pad results may be read in 1 minute.
 - After 2 minutes, reaction colors may fade.
 - None of the above.